Little Wave "Fl!p" XP

Little Wave "Fl!p" XPe

Little Wave "Fl!p" TTL

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Ki  Mobility

I. INTRODUCTION

Thank you for purchasing a Little Wave “Flip” XP or Little Wave “Flip” XPe wheelchair!

Please do not use this wheelchair without first reading this entire manual. BEFORE riding, you should be trained in the safe use of this chair by an Assistive Technology Practitioner (ATP) or clinical professional.

If you have any questions or concerns about any aspect of this wheelchair, this manual, or the service provided by us or your retail supplier, please do not hesitate to contact us by telephone at:

715-254-0991

In writing at:

Ki Mobility
5201 Woodward Drive
Stevens Point, WI 54481
U.S.A

Or via email at:

sales@kimobility.com

Or via our Authorized EU Representative:

James Leckey Design
19C Ballinderry Road
Lisburn
BT28 2SA
Phone: 0800 318265 (UK) or 1800 626020 (ROI)
www.leckey.com

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I.	INTRODUCTION	
II.	TABLE OF CONTENTS	
III.	NOTICE - READ BEFORE USE	
A.	Your Safety and Stability	4
IV.	WARNINGS	
A.	Signal Words.....	4
B.	General Warnings.....	5
C.	Positioning Belts	6
D.	Riding Your Wheelchair	7
E.	Power Drives.....	8
F.	Ascending Stairs	8
G.	Descending Stairs	8
H.	Transfers.....	9
I.	Your Wheelchair and the Environment.....	9
J.	Modifying your Wheelchair	10
K.	Wheelchair Stability	10
V.	SET UP & USE OF YOUR WHEELCHAIR	
A.	Your Little Wave "Flip" & It's Parts	13
B.	Transit Use.....	14
C.	Folding Wheelchair.....	17
D.	Height Adjustable T-Arm	19
E.	Angle Adjustable Locking Flip Up Extendable Armrest.....	21
F.	Armrest Warnings	22
G.	Swing Away Hangers	22
H.	Swing Away Hangers with 4-Way Latch.....	23
I.	Extension Tubes	24
J.	Elevating Leg Rests.....	25
K.	Pro Elevating Leg Rest	26
L.	Pediatric Pro Elevating Leg Rest.....	27
M.	Contracture Footrest	28
N.	Multi-Angle Footrest	29
O.	Backrest	30
P.	Reclining Backrest	32
Q.	Hand Tilt Mechanism	34
R.	Foot Tilt Mechanism.....	36
S.	Tilt Stops.....	36
T.	Seat Frame Setup	37
U.	Casters	38
V.	Axle Arm	40

II. TABLE OF CONTENTS

W.	Rear Wheels.....	42
X.	Drum Brake.....	44
Y.	Push / Pull to Lock Wheel Locks.....	46
Z.	Attendant Foot Lock (Hub Lock).....	47
AA.	Frame Width	48
BB.	Anti-Tips	50
CC.	Seat Pan.....	52
DD.	Bushings.....	53
EE.	Canopy.....	54
VI.	MAINTENANCE	
A.	Inspecting Your Wheelchair.....	55
B.	Routine Maintenance	56
C.	Cleaning.....	57
D.	Storage.....	57
VII.	TROUBLESHOOTING.....	58
VIII.	WARRANTY	60

III. NOTICE - READ BEFORE USE

A. Your Safety and Stability

Ki Mobility manufactures many different wheelchairs that might meet your needs. You should consult an Assistive Technology Professional when selecting which model would best meet your particular requirements and how the wheelchair should be set up and adjusted. Final selection of the type of wheelchair, options and adjustments rests solely with you and your medical professional. The options you choose and the set-up and adjustment of the wheelchair have a direct impact on its stability. Factors to consider that affect your safety and stability are:

- a. Your personal abilities and capabilities including strength, balance and coordination.
- b. The types of hazards and obstacles you might encounter during your day.
- c. The specific dimensions, options and set up. In particular, the seat height, seat depth, seat angle, back angle, size and position of the rear wheels and size and position of the front casters. Any change to any of these items will change the stability of your wheelchair. You should only make changes after consulting with a qualified professional.

IV. WARNINGS

A. Signal Words

Within this manual you will find what are referred to as “Signal” words. These words are used to identify and convey the severity of varying hazards. Before using this chair you, and each person who may assist you, should read this entire manual. Please note the Signal word and consider any warnings, cautions or dangers. Make sure to follow all instructions and use your chair safely. The Signal word refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in three main categories, as follows:

DANGER – Danger indicates an imminently hazardous situation which, if not avoided, will result in serious injury or death.

WARNING – Warning indicates a potentially hazardous situation which, if not avoided, could result in serious injury or death.

CAUTION – Caution indicates a potentially hazardous situation which, if not avoided, could result in injury or damage to your wheelchair.

These signal words will be placed throughout the manual, where appropriate to highlight the hazardous situation. Refer to the following list for hazardous situations that will apply to the general use of this wheelchair.

IV. WARNINGS

B. General Warnings

WARNING

WARNING: Do not exceed the weight limit of 165 lb (75 kg) for the Little Wave "Flip". This is the combined weight of user and all items carried. Exceeding the weight limit can cause damage to your chair or increase the likelihood of a fall or tip back resulting in severe injury or death to the user or others.

DANGER

DANGER: Do not use this chair for weight training. The movement of the additional weight will alter the center of gravity of the wheelchair increasing the likelihood of a tip-over which can cause damage to your chair or cause severe injury or death to the user or others.

WARNING

WARNING: If your wheelchair is equipped with inflatable tires, make sure the tires have been inflated to the correct tire pressure as indicated on the side wall of the tire. Your wheelchair provider can determine if you have inflatable tires. Using your wheelchair without properly inflated tires can have an effect on the stability of the wheelchair causing it to tip over resulting in death or injury to the user or others.

DANGER

DANGER: Do not attempt to push your wheelchair up or down ramps or traverse across a slope of greater than 9 degrees. This is dangerous and increases the likelihood of a fall or tip back resulting in severe injury or death to the user or others.

WARNING

WARNING: Do not attempt to push your wheelchair up an incline that is slick or coated with ice, oil or water. This can cause an unstable situation resulting in death or injury to the user or others.

DANGER

DANGER: Do not lean over the side or back of the wheelchair to extend your reach. This may cause you to fall out of the wheelchair or the wheelchair to tip over resulting in injury or death.

IV. WARNINGS

B. General Warnings

DANGER

DANGER: Do not attempt to lift the wheelchair by holding on to removable parts such as the arms or footrests. Only lift the wheelchair by holding on to the frame. This may cause a fall or loss of control and result in serious injury or death.

CAUTION

CAUTION: Do not overtighten the bolts and hardware that attaches various components together on the frame. This could cause serious damage and affect the safety and durability of the wheelchair.

C. Positioning Belts

Positioning belts are designed to assist with proper positioning within the wheelchair. They are not designed as seat belts. Use positioning belts **ONLY** to help support the user's posture. Misuse of positioning belts may cause severe injury to or death of the user.

- Ensure the user does not slide underneath the positioning belt in the wheelchair seat. If this occurs, the user's breathing may be hampered causing death or serious injury.
- The positioning belt should have a snug fit; tight enough to hold their position, but not so tight as to restrict breathing. You should be able to slide your hand between the positioning belt and the user.
- **NEVER** Use Positioning Belts:
 - a. As a restraint. A restraint requires a doctor's order.
 - b. On a user who is unconscious.
 - c. As an occupant restraint in a motor vehicle. A positioning belt is not designed to replace a seat belt that is attached to the frame of a vehicle, which would be required of an effective seat belt. During a sudden stop, with the force of the stop, the user would be thrown forward. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.

DANGER

DANGER: Failure to comply with the instruction above could result in serious injury or death.

IV. WARNINGS

D. Riding Your Wheelchair

Your chair is designed for use on solid, flat surfaces such as concrete, asphalt and flooring. Use caution if you push your wheelchair on a wet or slick surface.

WARNING

WARNING: Do not push your chair in sand, loose soil or over rough terrain. This may cause a loss of stability and result in a fall or loss of control and cause serious injury or death.

DANGER

DANGER: In most states, wheelchairs are not legal for use on public roads. If you find you must push on a public road, be alert to the danger of motor vehicles. Use of a wheelchair on a public road can cause serious injury or death.

WARNING

WARNING: Obstacles and road hazards (such as potholes and broken pavement) can damage your chair and may cause a fall, tip-over or loss of control. Failure to comply with this instruction could result in serious injury or death.

DANGER

DANGER: Do not ride your wheelchair on an escalator. Use of a wheelchair on an escalator can cause serious injury or death.

To minimize these risks:

- 1) Keep a lookout for danger-scan the area well ahead of your chair as you ride.
- 2) Make sure the floor areas where you live and work are level and free of obstacles.
- 3) Remove or cover threshold strips between rooms.
- 4) Install a ramp at entry or exit doors. Make sure there is not a drop off at the bottom of the ramp.
- 5) To Help Correct Your Center of Balance:
 - a. Lean your upper body **FORWARD** slightly as you go **UP** over an obstacle.
 - b. Press your upper body **BACKWARD** as you go **DOWN** from a higher to a lower level.
- 6) If your chair has anti-tip tubes, lock them in place before you go UP over an obstacle.
- 7) Keep both of your hands on the handrims as you go over an obstacle.
- 8) Never push or pull on an object (such as furniture or a doorjamb) to propel your chair.
- 9) Do not operate your wheelchair on roads, streets or highways.
- 10) Do not attempt to push over obstacles without assistance.

IV. WARNINGS

E. Power Drives

Ki Mobility does not recommend the installation of power drive systems on any Little Wave "Flip" wheelchair.

Little Wave "Flip" wheelchairs have not been designed or tested as power wheelchairs. If you add a power drive system to a Little Wave "Flip" wheelchair, be sure the manufacturer of the power drive system has validated and approved the combination of the power drive system and Little Wave "Flip" wheelchair as safe and effective.

WARNING

WARNING: Use of a power drive system that has not been properly validated could result in serious injury or death.

F. Ascending Stairs

- Have at least two people, who have sufficient strength and skill to handle the weight of the user and wheelchair, assist when trying to go up a set of stairs in this wheelchair.
- Move the wheelchair and user backwards up the stairs.
- Position one person behind the user, one person in front. The person in front must hold onto a non-removable part of the wheelchair.
- The rear attendant tilts the chair back and they both lift together. Take one step at a time.
- This may require the anti-tips be flipped up or removed. Make sure the anti-tips are reattached or flipped back down before using the wheelchair.

DANGER

DANGER: Failure to comply with the instructions above could result in serious injury or death.

G. Descending Stairs

- When descending a set of stairs the user should be facing forward.
- A person behind the user, who has sufficient strength and skill to handle the weight of the user and the wheelchair, should tilt the chair backward and let the chair down the stairs one step at a time on the rear wheels.
- This may require that anti-tips be flipped up or removed. Make sure the anti-tips are reattached or flipped back down before using the wheelchair.

DANGER

DANGER: Failure to comply with the instructions above could result in serious injury or death.

IV. WARNINGS

H. Transfers

A transfer requires good balance and stability. You should receive training from your therapist before attempting to do a transfer on your own.

- Before transferring out of your wheelchair every caution should be taken to reduce the gap between the two surfaces.
- Engage the wheel locks to lock the rear wheels.
- Rotate the casters forward to increase the wheelbase of the wheelchair.
- Remove or swing away the footrests.
- Have someone assist you unless you are well experienced in transfers.

It is dangerous to transfer on your own. It requires good balance and agility. Be aware there is a point during every transfer when the wheelchair seat is not below you.

WARNING

WARNING: Failure to comply with the instructions above may cause a fall or loss of control, which may result in serious injury or death.

I. Your Wheelchair and the Environment

- Your wheelchair is made of many different materials including metal and fabric. Exposure to water or excessive moisture may cause the metal in the wheelchair to rust or corrode and the fabric to tear. Dry your chair as soon as possible if exposed to water.
- **DO NOT USE YOUR WHEELCHAIR IN A SHOWER, POOL OR BODY OF WATER.** This will cause your wheelchair to rust or corrode and eventually fail.
- Do not operate your wheelchair in sand. Sand can get into the wheel bearings and moving parts. This will cause damage and eventually will cause the wheelchair to fail.
- Make sure any ramp, slope or curb cut you may attempt to ride on is compliant with ADA guidelines. Riding across, up or down any slope that is too great may cause a loss of stability.

ADA Guidelines and more information about accessible design are available at:
<http://www.ada.gov/>

WARNING

WARNING: Failure to comply with the instructions above may cause a fall or loss of control, which may result in serious injury or death.

IV. WARNINGS

J. Modifying your Wheelchair

Your wheelchair was engineered and manufactured under strict design controls. An integral part of this process is ensuring the various components work together correctly; they have been tested to various standards to ensure quality and are approved to work together.

YOU SHOULD NOT CHANGE, ADD OR REMOVE COMPONENTS OR OTHERWISE MODIFY THIS WHEELCHAIR. NO ONE SHOULD MODIFY THIS WHEELCHAIR EXCEPT BY ASSEMBLING APPROVED OPTIONS. THERE ARE NO APPROVED OPTIONS THAT INVOLVE DRILLING OR CUTTING THE FRAME BY ANYONE OTHER THAN A TRAINED KI MOBILITY ASSOCIATE. Contact Ki Mobility or an authorized Ki Mobility supplier before adding any accessories or components not provided by Ki Mobility.

DANGER: Failure to comply to these instructions may cause the wheelchair to fail and result in serious injury or death.

K. Wheelchair Stability

To ensure proper stability of your wheelchair, you must make sure the center of gravity and the wheelchairs base of support is correct for your balance and abilities. Many factors can affect these two elements:

- Seat height
- Seat depth
- Back angle
- Seat Angle
- Size and position of rear wheels
- Size and position of front casters
- Any seating system components
- Tilt Position

Generally, the most important factor is the position of the rear wheels for rearward stability. There are other actions than can have an adverse effect on your stability. You should consult with your wheelchair provider and clinicians familiar with your needs and capabilities in determining how this affects your use.

WARNING: This chair is equipped with a system to vary the tilt angle of the seat frame. Make sure that the chair is stable throughout the range of tilt angle change. Failure to comply to the instruction above could result in serious injury or death.

WARNING: Moving the rear wheels forward increases the likelihood of the wheelchair tipping backwards. Make small adjustments and proceed slowly until you learn the new balance point of your wheelchair. Failure to comply with the instruction above could result in serious injury or death.

IV. WARNINGS

K. Wheelchair Stability

WARNING

WARNING: The farther rearward you place the front casters the greater the likelihood of the wheelchair tipping forwards. If possible, have your casters mounted forward and whenever doing a static activity which involves shifting your weight, rotate the casters forward to increase your wheel base. Failure to comply with this instruction above could result in serious injury or death.

WARNING

WARNING: Always have a qualified technician set up your wheelchair with the accessories you plan to use daily.

Changes to how you sit or changes in your weight require your chair to be readjusted by a qualified technician. Always use anti-tips while you acclimate to any changes in your chair set up. Failure to comply with the instruction above could result in serious injury or death.

WARNING

WARNING: Changes to your Center of Gravity during your daily activities may occur many times a day, changing and affecting the stability of your wheelchair. You should be aware of these activities and take precautions to minimize the risk of a fall. Failure to comply with the instruction above could result in serious injury or death.

WARNING

WARNING: Dressing in your wheelchair produces movements and momentary positions that can reduce stability. Ensure that your anti-tips are in place and rotate your casters forward. Failure to comply with the instruction above could result in serious injury or death.

WARNING

WARNING: Be very careful when reaching for objects if this movement requires you to shift in your seat. This changes your center of gravity. Ensure that your anti-tips are in place. Failure to comply with the instruction above could result in serious injury or death.

WARNING

WARNING: Pushing up an incline shifts your center of gravity rearward and can reduce stability. Ensure your anti-tips are in place. Failure to comply with the instruction above could result in serious injury or death.

IV. WARNINGS

K. Wheelchair Stability

WARNING

WARNING: If attempting a wheelie to get over a curb or obstacle, ensure your anti-tips are in place and lean forward. Do not attempt a wheelie unless you have been trained and always have an attendant behind you to provide assistance if needed. Failure to comply with the instruction above could result in serious injury or death.

WARNING

WARNING: Placing items on the back or front of your wheelchair, such as a backpack or briefcase, alters the balance and center of gravity of the wheelchair. Since the weight of these items can vary greatly at each use do not assume you are accustomed to the balance point. Failure to comply with the instruction above could result in serious injury or death.

BE AWARE THAT CARRYING HEAVY OBJECTS ON YOUR WHEELCHAIR CAN HAVE AN ADVERSE EFFECT ON THE BALANCE WHICH MAY CAUSE A TIP-OVER WHICH MAY RESULT IN SERIOUS INJURY OR DEATH TO THE USER.

WARNING

WARNING: Ensure your anti-tips are in place. You should discuss how you plan to use your wheelchair or any changes you are planning with your clinician. Failure to comply with this instruction may create a potential hazardous situation which, if not avoided, could result in serious injury or death.


V. SET UP & USE OF YOUR WHEELCHAIR

A. Your Little Wave "Flip" & It's Parts

1. Inspect and maintain this chair. See *MAINTENANCE* on page 55.
2. If you detect a problem, make sure to service or repair the chair before use.
3. Have a complete inspection, safety check and service of your chair performed by an authorized supplier annually.

⚠ WARNING ⚠

WARNING: Failure to read or comply with these instructions may result in damage to your wheelchair, a fall, or loss of control causing severe injury to the user or others.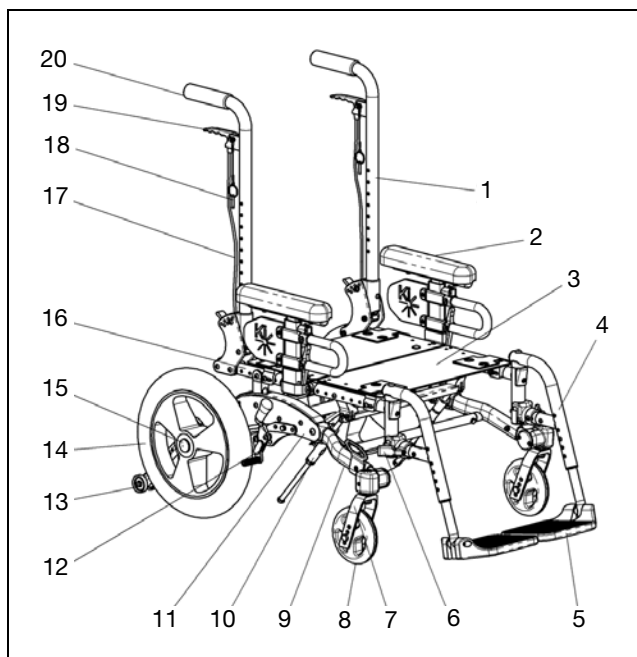

- | | |
|------------------------|------------------------|
| 1. Back Cane | 11. Base Frame |
| 2. Height Adj. Armrest | 12. Wheel Lock |
| 3. Seat Pan | 13. Anti-Tip |
| 4. Footrest Hanger | 14. Rear Wheel |
| 5. Footrest | 15. Quick Release Axle |
| 6. Swing Away Lever | 16. Seat Tube |
| 7. Caster Fork | 17. Tilt Cable |
| 8. Caster Wheel | 18. Tilt Locking Pin |
| 9. Caster Arm | 19. Tilt Lever |
| 10. Mech Lock | 20. Push Handle |

V. SET UP & USE OF YOUR WHEELCHAIR

B. Transit Use

It is always safest to transfer out of your wheelchair onto a seat in a motor vehicle with appropriate seat and shoulder belts. **Never use this wheelchair as a seat in a motor vehicle unless it has been equipped with the Transit Option.**

The Little Wave "Flip" Series wheelchair equipped with the Transit Option has been tested to and passed the **RESNA WC-4:2012, Section 19: Wheelchairs used as seats in motor vehicles and ISO 7176-19:2008 Wheelchairs -- Part 19: Wheeled mobility devices for use as seats in motor vehicles**. RESNA and ISO standards are designed to test the structural integrity of the wheelchair as a seat for use in a motor vehicle. These standards are also designed to create compatibility with Wheelchair Tie-down and Occupant Restraint Systems (WTORS).

Not all configurations of the Little Wave "Flip" wheelchairs are compatible with the Transit Option. Ki Mobility manages the configuration and does not offer the Little Wave "Flip" wheelchair except in compatible configurations. If you make changes to your Little Wave "Flip" wheelchair after you receive it, you should contact your wheelchair provider or Ki Mobility to make sure it is appropriate to continue to use your wheelchair as a seat in a motor vehicle.

Aftermarket seating may have replaced the original equipment seat and back support designed and tested as part of the Transit Option. Your wheelchair provider should tell you if the seating they provided is original equipment or replacement aftermarket seating. A complete system of wheelchair frame, seating, **Wheelchair Tie-down** and **Occupant Restraint Systems** and a properly equipped motor vehicle, that have all complied with the standards mentioned in this section, should be in place before using a Little Wave "Flip" wheelchair equipped with the Transit Option as a seat in a motor vehicle.


When using your wheelchair as a seat in a motor vehicle you should always observe the following instructions:

- The rider must be in a forward-facing position.
- The rider and all items carried must not weigh more than 165 lbs (75 kg).
- Backpacks and pouches should be removed and secured separately in the motor vehicle. In the event of an accident these items can become dangerous projectiles, which may injure or kill you or other occupants of the motor vehicle.
- The rider must use a Wheelchair Tie-down and Occupant Restraint System that complies with **RESNA WC-4:2012, Section 18: Wheelchair tie-down and occupant restraint systems for use in motor vehicles or ISO 10542-1:2012 Technical systems and aids for disabled or handicapped persons -- Wheelchair tie-down and occupant-restraint systems -- Part 1: Requirements and test methods for all systems**.
- Attach wheelchair tie-downs to the four securement points (two front, two rear) on the Little Wave "Flip" wheelchair with the Transit Option (Fig. 1) in accordance with the wheelchair tie-down manufacturer's instructions and **RESNA WC-4:2012, Section 18 or ISO 10542-1:2012 - Part 1**.
- Attach occupant restraints in accordance with the occupant restraint manufacturer's instructions and **RESNA WC-4:2012, Section 18 or ISO 10542-1:2012, Part 1**.

V. SET UP & USE OF YOUR WHEELCHAIR

B. Transit Use

- Use of lap belts, chest straps, shoulder harnesses, any other positioning strap system or positioning accessory should not be used, or relied on as an occupant restraint, unless it is marked as such by the manufacturer in accordance with **RESNA WC-4:2012, Section 18 or ISO 10542-1:2012, Part 1**.
- Use of headrests, lateral supports or other positioning accessories should not be used, or relied on as an occupant restraint, unless it is marked as such by the manufacturer in accordance with **RESNA WC-4:2012, Section 18 or ISO 10542-1:2012, Part 1 or RESNA WC-4:2012, Section 20: Wheelchair seating systems for use in motor vehicles or ISO 16840-4:2009 Wheelchair seating -- Part 4: Seating systems for use in motor vehicles**.
- Any aftermarket seating should be tested to comply with **RESNA WC-4:2012, Section 20 or ISO 16840-4:2009 - Part 4**.
- Attach the seating to the wheelchair frame in accordance with the seating manufacturer's instructions and **RESNA WC-4:2012, Section 20 or ISO 16840-4:2009 - Part 4**.
- Aftermarket accessories such as trays, oxygen tank holders, oxygen tanks, IV poles, back packs, pouches and other items not manufactured by Ki Mobility should be removed and secured separately in the motor vehicle. In the event of an accident, these items can become dangerous projectiles which may injure or kill you or other occupants of the motor vehicle.
- If the wheelchair has been involved in an accident, you should not continue to use it, as it may have suffered fatigue that may not be visible.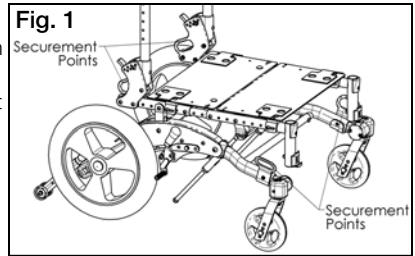

⚠ DANGER ⚠

DANGER: Failure to comply with any of these instructions could result in severe injury or death!

- When using this wheelchair as a seat in a motor vehicle you must remove any items attached to the vent tray, battery tray or oxygen tank holder and properly secure them separately.

⚠ DANGER ⚠

DANGER: Failure to remove items attached to the vent tray, battery tray or oxygen tank holder and properly secure them separately in the motor vehicle can result in these items becoming dangerous projectiles in the event of an accident. Oxygen tanks contain a highly pressurized gas that vigorously accelerates combustion. Failure to heed this warning can result in severe injury or death.

- If the chair is equipped with an oxygen tank holder, never use the wheelchair as a seat in a motor vehicle.

⚠ DANGER ⚠

DANGER: Using a Flip wheelchair equipped with an oxygen tank holder as a seat in a motor vehicle can result in severe injury or death.

V. SET UP & USE OF YOUR WHEELCHAIR

B. Transit Use

NOTE: To obtain copies of RESNA or ISO standards please contact the standards organizations below:

RESNA

1700 North Moore St., Suite 1540
Arlington, VA 22209
Phone: 703-524-6686
Fax: 703-524-6630
Email: technicalstandards@resna.org

ANSI/RESNA Standards:

RESNA WC-4:2012, Section 18:

Wheelchair tie-down and occupant restraint systems for use in motor vehicles.

RESNA WC-4:2012, Section 19:

Wheelchairs used as seats in motor vehicles.

RESNA WC-4:2012, Section 20:

Wheelchair seating systems for use in motor vehicles.

International Organization for Standardization (ISO)

BIBC II
Chemin de Blandonnet 8
CP 401
1214 Vernier, Geneva
Switzerland
Phone: +41 22 749 01 11
Fax: +41 22 733 34 30
Email: central@iso.org

ISO Standards:

ISO 10542-1:2012 Technical systems and aids for disabled or handicapped person --

Wheelchair tie-down and occupant-restraint systems -- Part 1:

Requirements and test methods for all systems.

ISO 16840-4:2009 Wheelchair seating -- Part 4:

Seating systems for use in motor vehicles.



ISO 7176-19:2008 Wheelchairs -- Part 19:

Wheeled mobility devices for use as seats in motor vehicles.



V. SET UP & USE OF YOUR WHEELCHAIR

C. Folding Wheelchair

1. Remove footrest and "T" armrests (See Sections D: Height Adjustable T-Arm and G: Swing Away Hangers for removal instructions).
2. Remove seating as necessary.
3. Using tilt hand levers (Section Q: Hand Tilt Mechanism) or foot tilt pedal (Section R: Foot Tilt Mechanism), tilt front of seat frame down, until bottom of seat frame contacts base top of base frame cover (Fig. 2). Make sure mech locks are retracted properly (Fig. 3).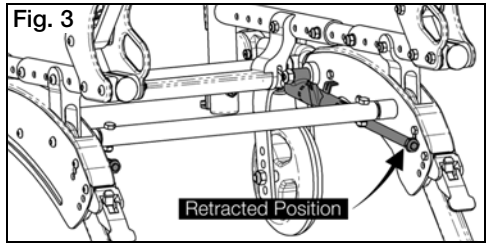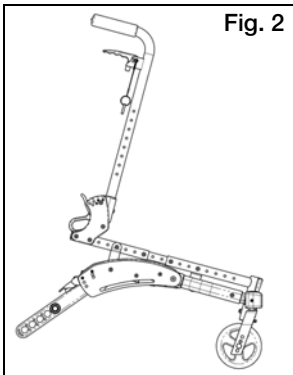


4. Push both backrest release levers forward and unlatch (Fig. 4).
5. Fold back canes downward toward front of seat frame (Fig. 5).
6. If height adjustable backrest is installed, push handles can be folded down also to minimize folded size.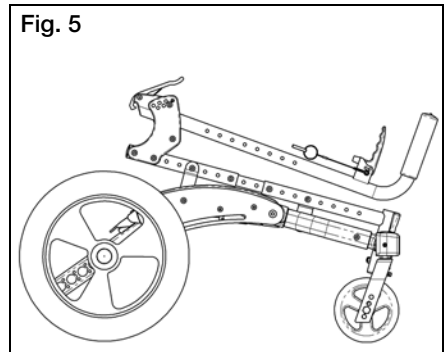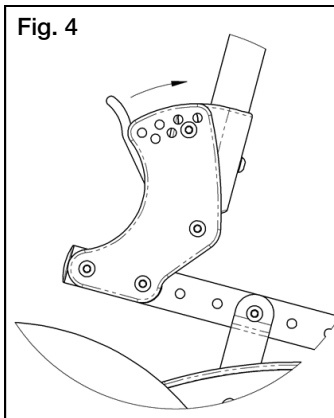


V. SET UP & USE OF YOUR WHEELCHAIR

C. Folding Wheelchair

7. Flip anti-tips up (See Section BB: Anti-Tips), then remove wheels and flip up red levers on top of axle arms on both sides of chair (Fig. 6). Levers will remain upright until chair is folded.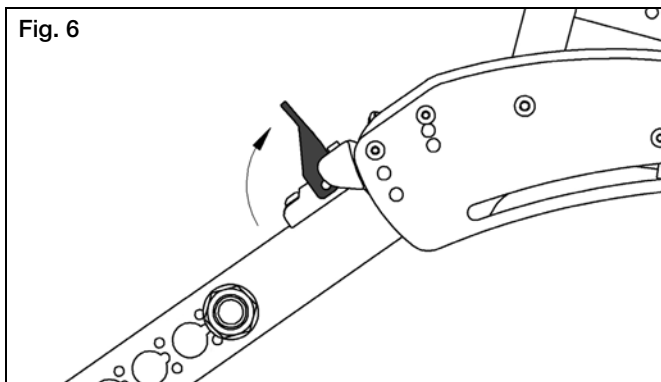

8. Lift rear of chair off of the floor and swing axle bar down and forward as far as it will go, engaging into slit in bottom of base frame between side plates (Fig. 7).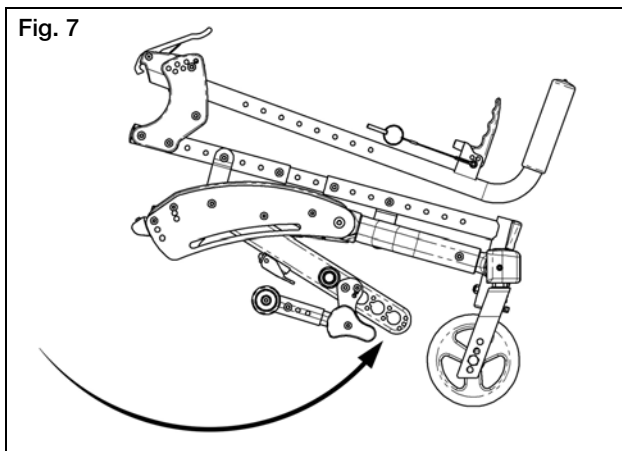

V. SET UP & USE OF YOUR WHEELCHAIR

D. Height Adjustable T-Arm

1. Installation


- a. Slide the outer arm post into the receiver mounted to the wheelchair frame.
- b. The armrest will automatically lock into place. Check to make sure the locking lever is as shown (Fig. 8:B).

2. Height Adjustment

- a. Rotate release lever (Fig. 8:A).
- b. Slide armrest pad up or down to desired height.
- c. Return lever to locked position against arm post.
- d. Push arm pad until upper arm post locks firmly into place. Check to make sure the locking lever is as shown (Fig. 8:A).

3. Removal

- a. Squeeze release lever (Fig. 8:B) and remove the armrest.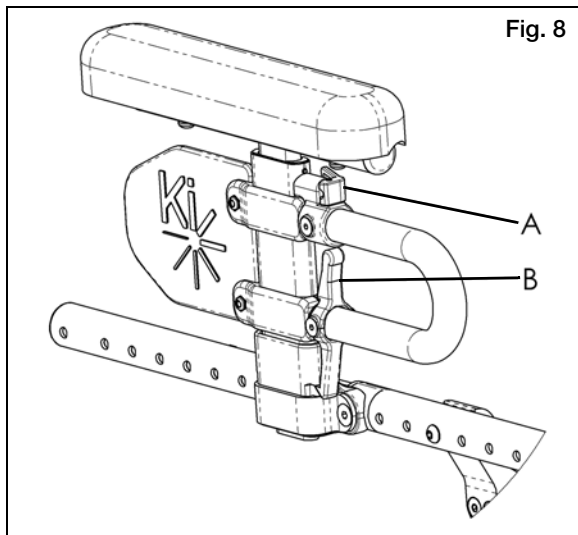


V. SET UP & USE OF YOUR WHEELCHAIR

D. Height Adjustable T-Arm

4. Adjusting Position

- Remove both screws (Fig. 9:A) from each armrest receiver (Fig. 9:B), retaining the spacers (Fig. 9:C) if installed. If not installed at factory, they are included in a separate bag in the packaging.
- Move receiver to desired location on seat frame and reinstall screws with washers and nuts to tighten.
- If screws for new armrest receiver location coincide with seat frame screws (Fig. 9:D and E), remove seat frame screws and replace with armrest receiver screws. Retain (do not discard) these screws for potential future use, or optionally, they may be retained by reinstalling them in an alternate location on the seat frame.
- If one of the armrest receiver screws lies on the center growth section of the seat frame, be sure to use the spacers (as described in step 1) around growth tube in location of mounting hole.

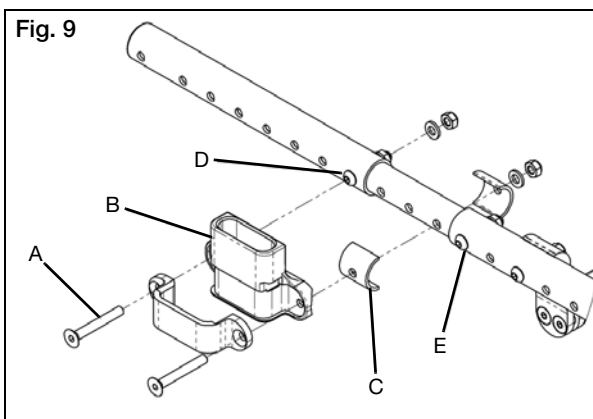DANGER: Failure to comply with the instructions above may result in the armrest accidentally disconnecting from the wheelchair and result in a fall or loss of control and may cause serious injury or death.

DANGER: Never attempt to lift the chair by the armrests; they may break or disconnect resulting in a fall or loss of control and may cause serious injury or death.

V. SET UP & USE OF YOUR WHEELCHAIR


E. Angle Adjustable Locking Flip Up Extendable Armrest

Adjustments

1. Set the angle of the armrest based on your preferences. There are five holes that can be used to set the angle (Fig. 10:A). Tighten the bolt once angle is set.
2. Set the length of the armrest based on your preferences. To adjust the length, remove the bolts and spacer (Fig. 10:B) on the tube and the screw (Fig. 10:C) closest to the back of the chair. Slide the armrest to desired length available by the predrilled holes and reinstall the screw and bolts.
3. Set the height of the armrest on the chair based on your preferences. There are four holes on the armrest that allow for two different height settings for each set of holes on the back tube. Use the holes that provide the correct height setting for the user. The two bolts (Fig. 10:D) pass through the spacer (Fig. 10:E), sleeves (Fig. 10:F), back posts and into the armrest.

Use

1. Press the release lever (Fig. 10:G) up to release the armrest and swing it upwards. To return the armrest to the operating position, push the armrest down until the lever clicks and locks.

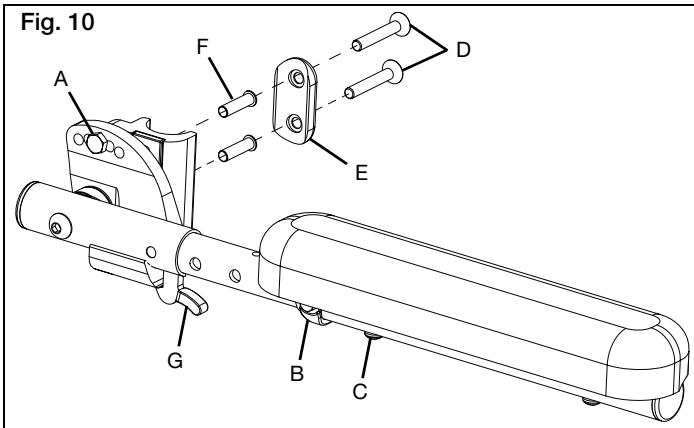DANGER: Always make sure the arms are locked in place before using them for repositioning yourself. Failure to do so may result in a fall or loss of control and may cause serious injury or death.

DANGER: Never attempt to lift the chair by the armrests; they may break or disconnect, resulting in a fall or loss of control and may cause serious injury or death.

V. SET UP & USE OF YOUR WHEELCHAIR

F. Armrest Warnings

- All Ki Mobility armrests are designed to detach from the chair and will not bear even the weight of this chair.
- NEVER lift this chair by its armrests. The armrests will release and the user may fall.
- Lift this chair only by non-detachable parts of the main frame.

⚠ WARNING ⚠

WARNING: Failure to heed these instructions may result in a fall, tip-over or loss of control causing severe injury to the user or others.

G. Swing Away Hangers

1. Installation

- a. Place swing away pivot saddle into the receiver on front frame tube with the footrest facing either inward or outward from the frame (Fig. 11).
- b. Rotate the footrest so that it aligns with the frame until it locks into place in the latch block (Fig. 12).

2. Swinging the Footrest Away

- a. Push release latch toward the frame.
- b. Rotate footrest outward or inward as desired.

3. Removal

- a. To remove footrest, push release latch toward the frame.
- b. Lift the footrest straight upward to remove. You may also swing the footrest inward or outward before lifting it off.

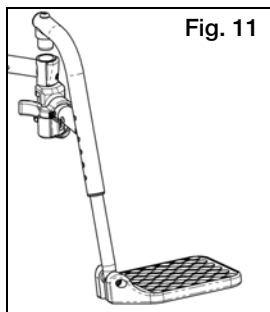
Fig. 11

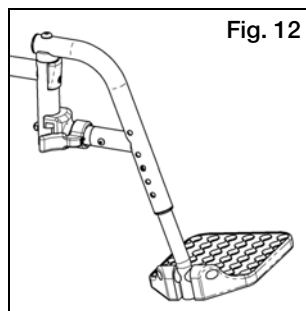
Fig. 12

⚠ WARNING ⚠

WARNING: Always make sure the hangers are locked in place before using them or riding the wheelchair. Failure to do so may result in a fall or loss of control and may cause serious injury or death.

⚠ DANGER ⚠

DANGER: Never attempt to lift the chair by the hangers; they may break or disconnect resulting in a fall or loss of control and may cause serious injury or death.

V. SET UP & USE OF YOUR WHEELCHAIR

H. Swing Away Hangers with 4-Way Latch

1. Installation

- Place swing away pivot saddle into the receiver on front frame tube (Fig. 13:A).
- Rotate the footrest so that it aligns with the frame and locks into place in the latch block (Fig. 13:B).

2. Swinging the Footrest Away

- Push or pull on release latch.
- Rotate footrest outward or inward as desired.

3. Removal

- To remove footrest, push or pull release latch.
- Lift the footrest straight upward to remove. You may also swing the footrest inward or outward before lifting it off.

4. Rotating (Fig. 14)

The 4-way latch has eight possible configurations, four with the curve of the lever facing outward and four with the curve of the lever facing inward. See Fig. 15 for possible configurations.

- To rotate the 4-way latch, remove the screw using a 3mm Allen wrench while the hanger is still on the chair (spring must be engaged to remove and reinsert screw and keeping the hanger on the chair keeps the spring engaged). Ensure nut does not fall out.
- Rotate the 4-way latch to desired orientation and reinsert screw with 3mm Allen wrench. Ensure that nut stays in position while tightening the screw. Do not overtighten screw or mechanism will bind.

NOTE: To reverse the 4-way latch, the same screw is removed, but the hanger has to be removed from the latch block. Once removed, slide the latch off, flip over and reinstall. Ensure spring is engaged, by pushing and holding the latch button in, and nut stays in position while reinstalling the screw.

NOTE: In-line position is not achievable with the Pro ELR Footrest option.

NOTE: On the Flip wheelchair, using the 4-way latch in the vertical in-line position allows the chair to be folded without removing hangers.

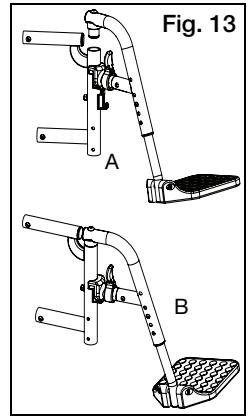
Fig. 13

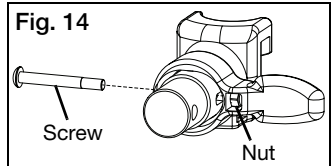
Fig. 14

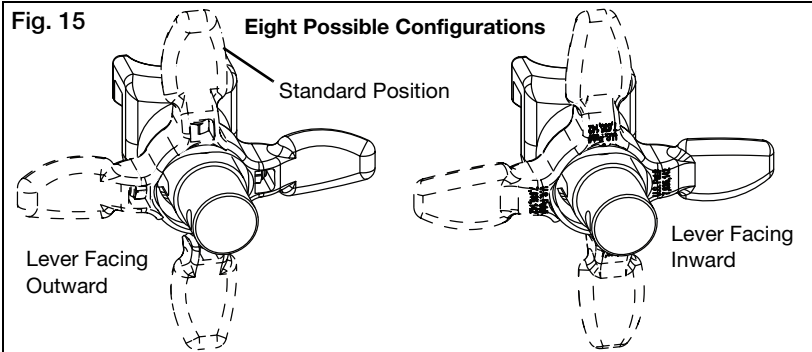
Fig. 15

Eight Possible Configurations

Standard Position

Lever Facing Outward

Lever Facing Inward

V. SET UP & USE OF YOUR WHEELCHAIR

I. Extension Tubes



1. Adjustment

- Remove mounting fasteners from each side of the hanger tube.
- Slide footrest extension tube to the desired height.
- Line up holes and reassemble fasteners in desired hole through hanger and extension tube.
- Follow same procedure on opposite side (Fig. 16).

At the lowest point, footrests should be **AT LEAST 2 ½ INCHES (6.35 cm)** off the ground. If set too low, they may "catch" on obstacles you would expect to find in normal use. This may cause the chair to stop suddenly and tip forward (Fig. 17).

To Avoid a Trip or Fall When You Transfer:

- Make sure your feet do not get caught in the space between the footrests.
- Avoid putting weight on the footrests as the chair may tip forward.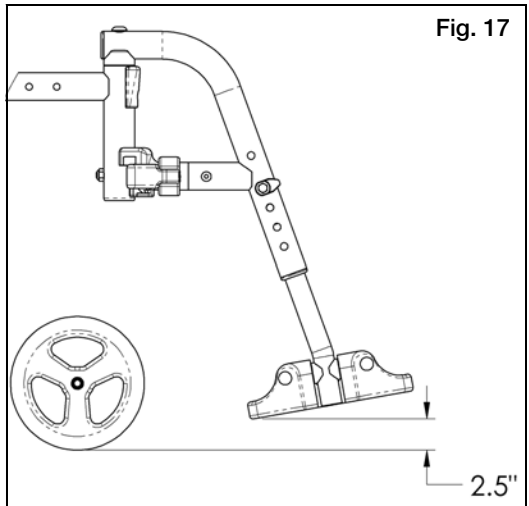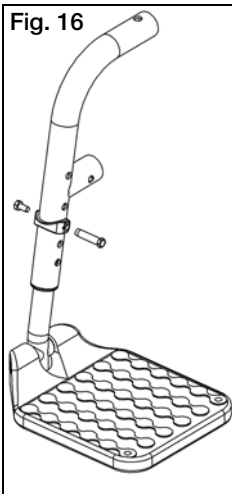

V. SET UP & USE OF YOUR WHEELCHAIR

J. Elevating Leg Rests

1. Installation

- Place swing away pivot saddle into the receiver on front frame tube with the footrest facing either inward or outward from the frame. Similar to swing away footrest pictured in Fig. 11 and 12.
- Rotate the footrest so it aligns with the frame until it locks into place in the latch block.

2. Removal

- To remove footrest, push release latch toward the frame.
- Rotate footrest outward and lift.

3. Extension Tube Adjustment

- Remove mounting bolt that is threaded into the leg rest tube in the center of the pad bracket (Fig. 18:C). In some circumstances this bolt may be inserted in the tube below the pivot bracket.
- Slide footrest extension tube to the desired height.
- Line up holes and replace the bolt through leg rest tube and extension tube.
- Follow same procedure on opposite side.

4. Angle Adjustment

- To raise, lift up on the rear of the extension tube (Fig. 18:A). The rod will slide through the ratchet in this direction. Stop at desired position.
- To lower, firmly hold the leg from behind the extension tube (Fig. 18:A). Pull forward on the lever (Fig. 18:B) and while holding the lever, lift the leg rest. Releasing the lever will cause the leg rest to lock into position.

To Avoid a Trip or Fall When You Transfer:

- Make sure your feet do not get caught in the space between the footrests.
- Avoid putting weight on the footrests as the chair may tip forward.

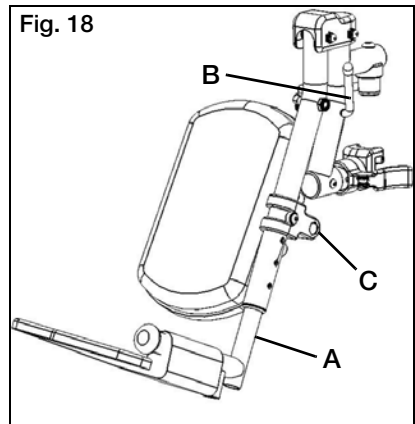
Fig. 18

⚠ WARNING ⚠

WARNING: At the lowest point, footrests should be **AT LEAST 2 ½ INCHES (6.35 cm)** off the ground. If set too low, they may "catch" on obstacles you would expect to find in normal use. This may cause the chair to stop suddenly and tip forward and could result in a fall or loss of control and may cause serious injury or death.

⚠ DANGER ⚠

DANGER: Never attempt to lift the chair by the hangers; they may break or disconnect resulting in a fall or loss of control and may cause serious injury or death.

V. SET UP & USE OF YOUR WHEELCHAIR

K. Pro Elevating Leg Rest

Installation

NOTE: The Pro ELR is mounted onto the chair the same way as a swing away hanger.


1. Place swing away pivot saddle into the receiver on front frame tube with the leg rest facing either inward or outward from the frame.
2. Rotate the leg rest so that it aligns with the frame and locks into place in the latch block.

Adjusting Height of Calf Pad

1. Loosen nut (Fig. 19:A) using a 10mm wrench.
2. Slide calf pad arm up or down to desired location. Retighten nut.


Adjusting Depth of Calf Pad

1. Remove screw (Fig. 19:B) and nut (Fig. 19:C) on calf pad arm using a 5mm Allen wrench and a 13mm wrench.
2. Pick the desired location based on the four predrilled holes and reinstall screw and nut.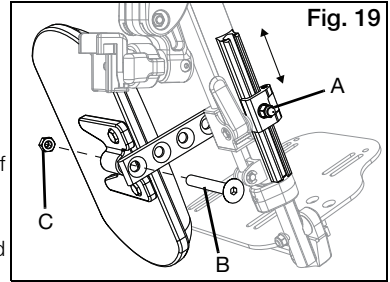


Adjusting Length of Footrest

1. Loosen set screw (Fig. 20:A) with a 4mm Allen wrench.
2. Slide extension tube in or out to get to desired length and secure by tightening set screw.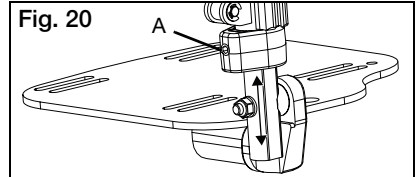

Adjusting Knee Height

1. Using two 10mm socket wrenches, loosen the two nuts (Fig. 21:A) on the cover.
2. Adjust knee height to desired setting.
3. Retighten the two nuts (Fig. 21:A) after desired height is attained.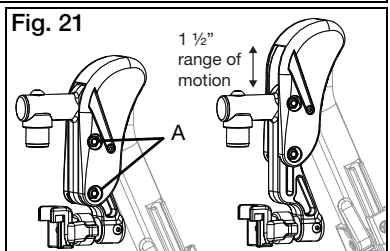

Use


1. To raise the Pro ELR, lift the leg rest tube (Fig. 22:A) to desired angle of elevation.
2. To lower the Pro ELR, press and hold the lever lock (Fig. 22:B) while pushing the leg rest tube (Fig. 22:A) down.

NOTE: Remove the Pro ELR from the chair or remove weight from the Pro ELR while lowering to avoid a sudden drop when the lever lock is pushed.

NOTE: The calf pad can swing outward to clear the front of the chair for transfers.

Removal

1. To remove leg rest, push or pull release latch.
2. Lift the leg rest straight upward to remove. You may also swing the leg rest inward or outward before lifting it off.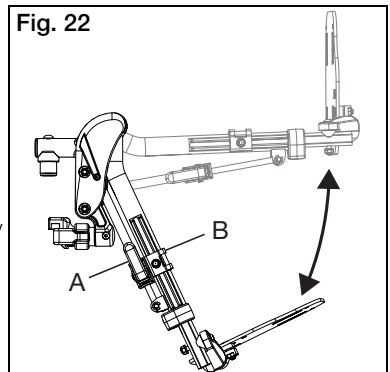

V. SET UP & USE OF YOUR WHEELCHAIR

L. Pediatric Pro Elevating Leg Rest

Installation

NOTE: The Pediatric Pro ELR is mounted onto the chair the same way as a swing away hanger.

1. Place swing away pivot saddle into the receiver on front frame tube with the leg rest facing either inward or outward from the frame.
2. Rotate the leg rest so that it aligns with the frame and locks into place in the latch block.


Adjusting Height of Calf Pad

1. Loosen nut (Fig. 23:D) using a 10mm wrench.
2. Slide calf pad arm up or down to desired location. Retighten nut.

Adjusting Depth of Calf Pad

1. Remove screw (Fig. 23:C), washer (Fig. 23:B) and nut (Fig. 23:A) on calf pad arm using a 5mm Allen wrench and a 13mm wrench.
2. Pick the desired location based on the four predrilled holes and reinstall screw and nut.


Fig. 23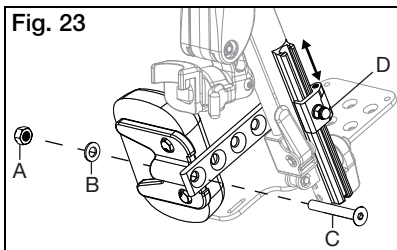

Adjusting Knee Height

1. Using two 10mm socket wrenches, loosen the two nuts (Fig. 24:A) on the cover.
2. Adjust knee height to desired setting.
3. Retighten the two nuts (Fig. 24:A) after desired height is attained.

Fig. 24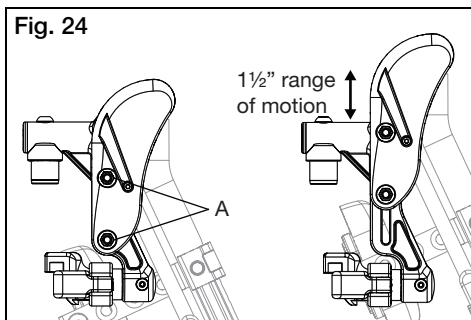

Use

1. To raise the Pediatric Pro ELR, lift the leg rest tube (Fig. 25:A) to desired angle of elevation.
2. To lower the Pediatric Pro ELR, press and hold the lever lock (Fig. 25:B) while pushing the leg rest tube (Fig. 25:A) down.


NOTE: Remove the Pediatric Pro ELR from the chair or remove weight from the Pediatric Pro ELR while lowering to avoid a sudden drop when the lever lock is pushed.

NOTE: The calf pad can swing outward to clear the front of the chair for transfers.

Removal

1. To remove leg rest, push or pull release latch.
2. Lift the leg rest straight upward to remove. You may also swing the leg rest inward or outward before lifting it off.

Fig. 25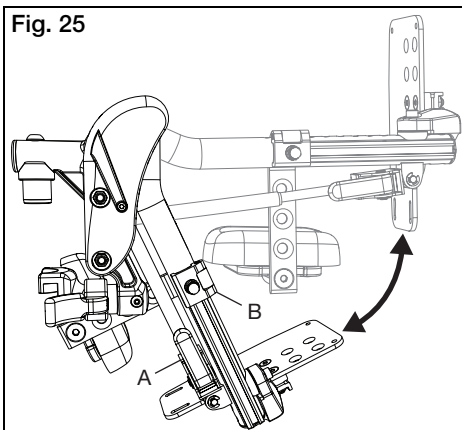


V. SET UP & USE OF YOUR WHEELCHAIR

M. Contracture Footrest

1. Changing Footrest and Footplate Angle


NOTE: Contracture Footrest is shown in the outside mount configuration. An inside mount can be achieved by moving the leg rests to the opposite side.

- The footrests on a contracture footrest can be angled toward the chair and away from the chair by simply adjusting to upper pivot point (Fig. 26:A).
- The footplates on a contracture footrest can also be moved front to rear (Fig. 26:B), are angle adjustable (Fig. 26:C) and can be flipped up for transfer (Fig. 26:D).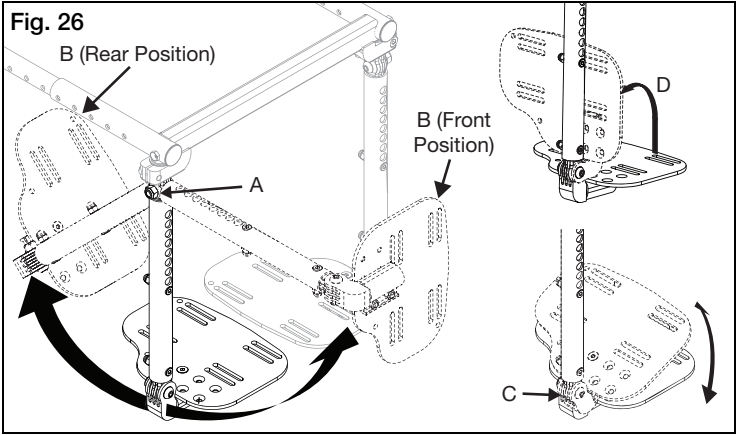

2. Changing Footrest Height

- Remove screw (Fig. 27:B) and nut (Fig. 27:C) from the long pivot (Fig. 27:A) and slide extension tube to desired height. See Chart in Fig. 27 for footrest heights that correspond with each hole.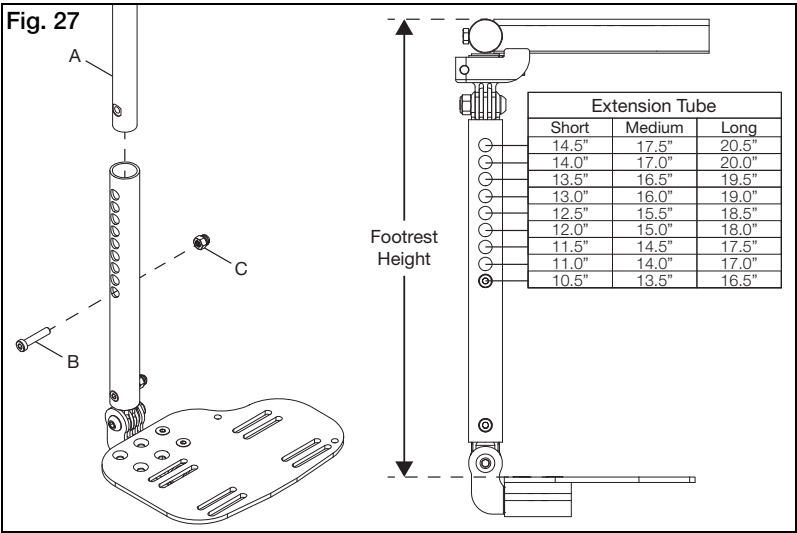


V. SET UP & USE OF YOUR WHEELCHAIR

N. Multi-Angle Footrest

1. Changing Footrest Angle (Fig. 28)


The Multi-Angle Footrest utilizes a ball joint which allows the footrest to be set in a variety of angles just by applying pressure to different areas on the top of the footplate.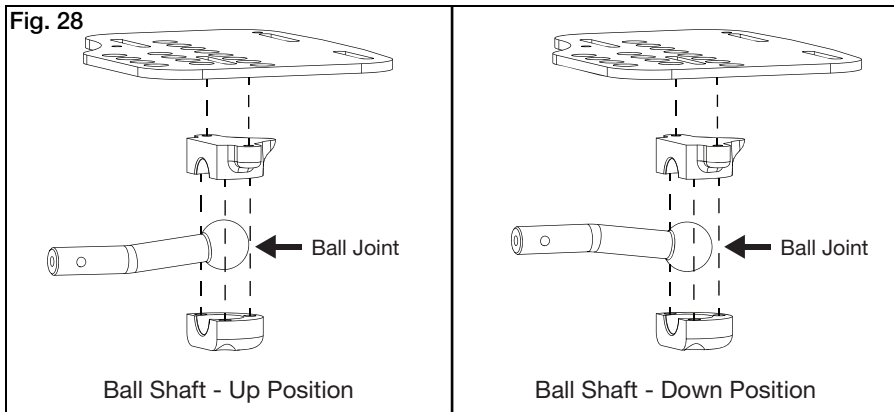

2. Changing Footrest Height

a. Remove footplate assembly from hanger by removing screw (Fig. 29:G), washer (Fig. 29:F), washer (Fig. 29:E), saddle spacer (Fig. 29:D), washer (Fig. 29:C), washer (Fig. 29:B) and nut (Fig. 29:A) using a 5mm Allen wrench and a 10mm wrench.

a. Slide the footplate assembly to desired height and reinstall hardware in nearest hole.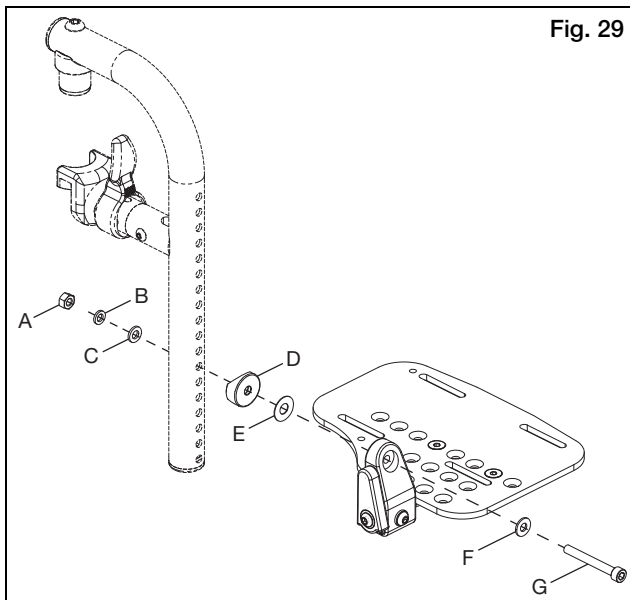


V. SET UP & USE OF YOUR WHEELCHAIR

O. Backrest



1. Folding Backrest Down

- If chair is equipped with a fold down backrest, push levers (Fig. 30:A) toward the front of the chair to disengage the latches. If backrest seating is attached, both latches must be disengaged simultaneously in order to allow the backrest to fold.
- Push forward on the backrest to rotate it downward. Once it begins to rotate forward, the latch levers may be released.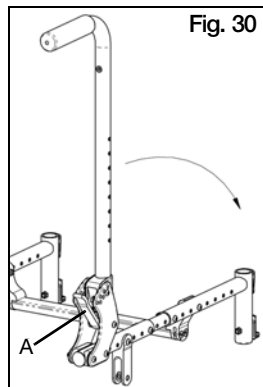

2. Angle Adjustment

- Using a 4mm Allen wrench and a 10mm wrench, remove button head screws (Fig. 31:A) with their washers and nuts.
- Rotate backrest to desired angle. Each hole represents 5° of rotation. The upper row of holes correspond to -5°, 5°, 15° and 25°, and the lower row of holes correspond to 0° (vertical), 10° and 20° (See Fig. 32).
- Replace button head screw and tighten with washer and nut.
- Repeat for opposite side of chair.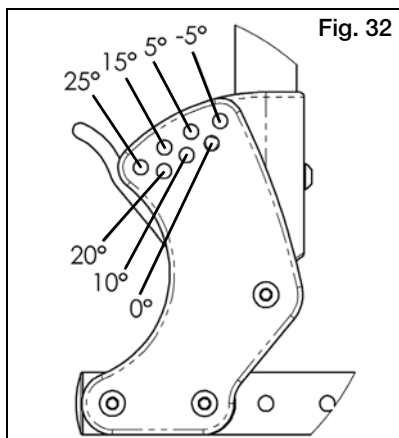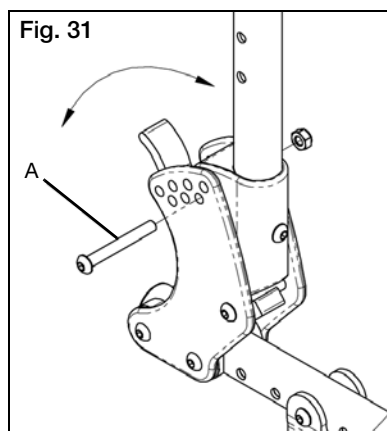

V. SET UP & USE OF YOUR WHEELCHAIR


O. Backrest

3. Adjusting Height

- If chair is equipped with a height adjustable backrest, remove both screws (Fig. 33:A) of the telescoping backrest tubes using a 10mm wrench.
- Move upper tubes up or down to set height as desired.
- Replace screws and nuts to set height.


4. Push Handle Angle Adjustment

- If chair is equipped with a height adjustable backrest, push buttons of locking hinges (Fig. 33:B). Both buttons must be pressed simultaneously to adjust angle.
- While holding buttons down, rotate push handle to desired position.
- Release buttons when desired angle is achieved.
- Verify that hinge is locked by pulling upwards on push handles.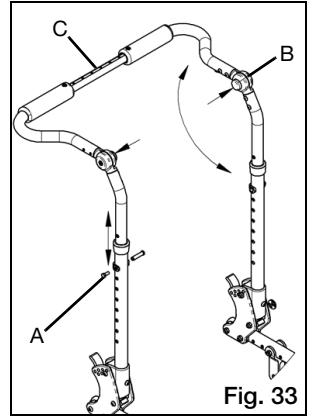

5. Removable Stroller Handle

- Depress buttons (Fig. 34:A) on both sides of chair toward lower end of stroller tubes to disengage locking tabs.
- Pull handle out of receivers to remove or push stroller handle into receivers to install. For best results, insert both sides at the same time.
- Verify snap buttons are engaged in receivers.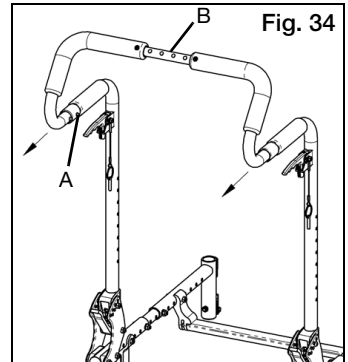

⚠ DANGER ⚠

DANGER: Do not attempt to lift or tilt the wheelchair by holding the center strut that joins the Adjustment Height Back push handles (Fig. 33:C) or the center strut that joins Removable Stroller Handle push handles (Fig. 34:B). These struts may break resulting in a fall or loss of control and may cause serious injury or death.

6. Dynamic Back

- See instructions for this back provided separately with chair.

V. SET UP & USE OF YOUR WHEELCHAIR

P. Reclining Backrest

WARNING

WARNING: Back height, back angle, seat depth, tilt angle, recline angle, seat height, size/position of rear wheels, size/position of front casters and any added weight to the back of the chair such as accessories, backpacks and oxygen tanks all relate to and affect the stability of the wheelchair. Any adjustments or change to one or more of the items listed above can decrease the stability of the wheelchair. Take caution when making changes to the above items. Consult a qualified technician when adjusting back height, back angle, seat depth, seat height, size/position of rear wheels and size/position of front casters. Inappropriate adjustments can result in serious injury.

Reclining Back Notes for Operation

1. After making any adjustments to the system make sure all attaching hardware is securely tightened.
2. Always ensure the wheelchair is stable in the fully reclined position.
3. In order to ensure maximum safety and security for the patient in the chair, it is important to make sure the patient is properly positioned before reclining or inclining (sitting up) the backrest.
4. Always engage the wheel locks while reclining or inclining to ensure maximum safety of the patient.
5. Before using the reclining backrest, ensure the anti-tips are adjusted properly to ensure the wheelchair will not become unstable when in use.

DANGER

DANGER: Tilting or reclining the chair beyond level greatly increases the risk of falling out of the chair which can result in serious injury or death.



V. SET UP & USE OF YOUR WHEELCHAIR

P. Reclining Backrest

Instructions for Use:

Recline / Incline:

1. Unlock the system prior to recline or incline adjustment by removing the locking pin from the the locking hole on each recline trigger (Fig. 35).
2. Ensure wheel locks are engaged and wheelchair is on a level surface.
3. Grip the handles securely and slowly depress both reclining triggers (Fig. 36) at the same time to release the gas cylinders. This will allow you to adjust the backrest position as follows:
 - a. **Reclining** - to open the angle of the backrest relative to the seat frame, or recline, by applying downward force on gas springs.
 - b. **Inclining** - return to upright position by applying force upward and pushing the backrest towards the front of the wheelchair.
4. Once desired backrest recline angle has been achieved, slowly release both reclining triggers.
5. To lock the system, insert the locking pin into the locking pin hole (Fig. 35). It is recommended that you always lock both reclining backrest triggers when you are done positioning the patient.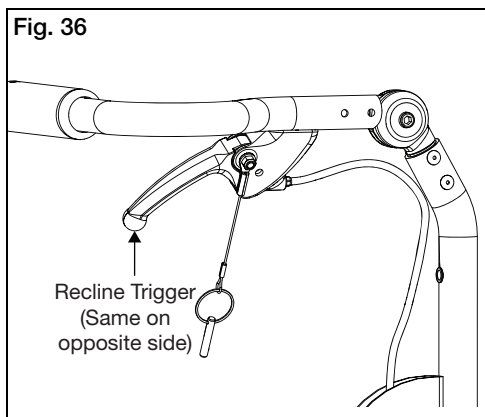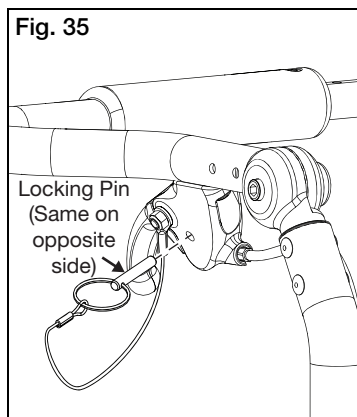

V. SET UP & USE OF YOUR WHEELCHAIR



Q. Hand Tilt Mechanism

1. Tilting Seat Frame

- Squeeze trigger (Fig. 37:A).
- While keeping triggers depressed, rotate seat frame to desired position.
- Release trigger to lock into place.

2. Cable Adjustment

- Unlock jam nut (Fig. 38:A) from cable adjuster body (Fig. 38:B) using a 10mm and 3/8" open ended wrench.
- Twist adjuster body in or out until slack in cable is eliminated, but also such that the mech lock lever (Fig. 38:C) is still fully open.
- Verify function of cable. Mech locks should slide freely when triggers are pulled and be fully locked when triggers are released.
- Once proper adjustment is obtained, tighten jam nut against adjuster body to lock.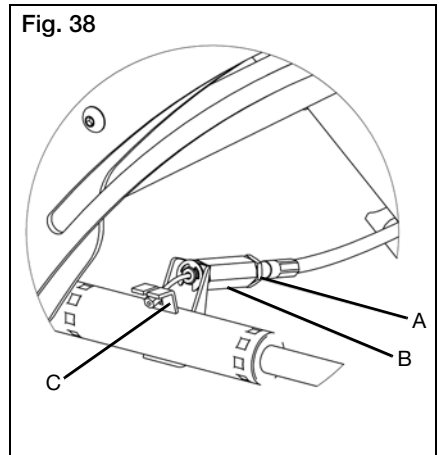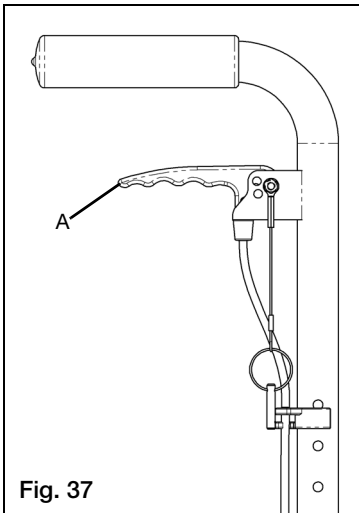

V. SET UP & USE OF YOUR WHEELCHAIR

Q. Hand Tilt Mechanism

3. Locking Triggers



- a. Pins are provided to secure the trigger in either the locked or unlocked position.

To secure the triggers in the locked position:

- a. Determine which style back canes and trigger were provided with the chair by checking against Fig. 39 and 40.
- b. Make sure the chair is locked by pushing on the back handles.
- c. Insert the pin attached to the trigger by a lanyard into hole (Fig. 39:A and 40:A).
- d. Make sure the trigger no longer releases the tilt locks by gently squeezing the trigger and pushing on the back canes to tilt the chair.

To secure the triggers in the unlocked position:

- a. Determine which style back canes and triggers were provided with the chair by checking against Fig. 39 and 40.
- b. Squeeze the triggers to unlock the tilt mechanism while holding the back canes and controlling the tilt position.
- c. Allow the seat frame to tilt until it comes to rest in a stable position.
- d. Insert the pin attached to the trigger by a lanyard into hole (Fig. 39:B and 40:B).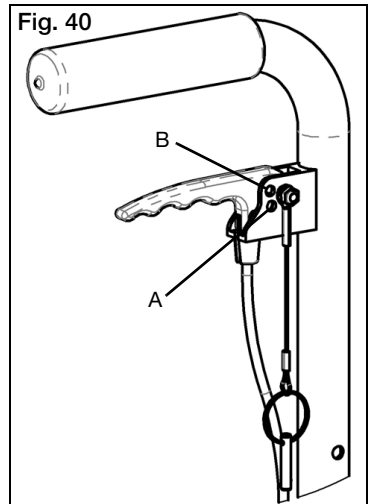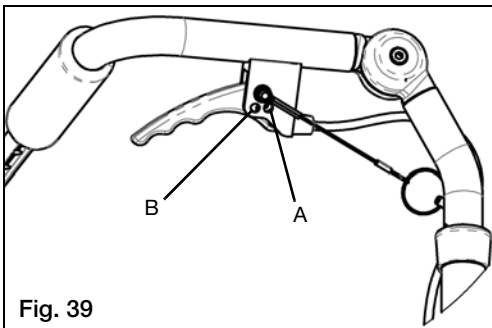

⚠ WARNING ⚠

WARNING: Never leave someone seated in the chair unattended with the triggers secured in the unlocked position. This may lead to serious injury to the person seated in the wheelchair.

V. SET UP & USE OF YOUR WHEELCHAIR

R. Foot Tilt Mechanism

1. Tilting Seat Frame

- Depress foot tilt pedal (Fig. 41:A).
- Tilt seat frame to desired position while keeping pedal depressed.
- Release pedal to lock into place.

2. Cable Adjustment

- Using two 10mm wrenches, loosen jam nut (Fig. 41:B) while holding adjuster (Fig. 41:C) in place. Repeat for opposite cable.
- Adjust cable by twisting adjusters until desired tension is obtained, ensuring that actuators on mech locks are not being actuated at all while pedal is released. Ensure that mech locks slide freely when pedal is depressed.
- Once desired tension is obtained, hold adjusters in place with wrench and tighten jam nut. Repeat for both cables.

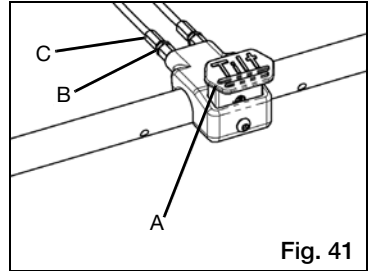
Fig. 41

S. Tilt Stops

1. Tilt Stop Adjustment

- Remove screws from tilt stops and install onto shaft of mech lock. Reinstall screws, but do not tighten (Fig. 42:A). Repeat for opposite mech lock.
- Set chair to desired angle that tilt would like to be limited to.
- For limiting rearward tilt, slide tilt stop to bottom of mech lock body and tighten into place.
- For limiting forward tilt, tilt stops may be placed above the body of the mech lock (Fig. 42:B). When desired angle is achieved, slide tilt stop against top of mech lock body and tighten.
- Ensure both tilt stops are against their respective mech locks simultaneously.

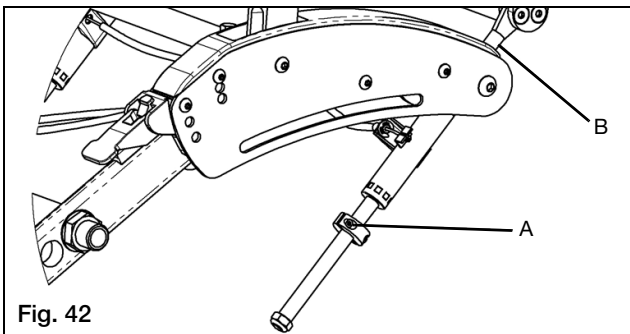
Fig. 42

V. SET UP & USE OF YOUR WHEELCHAIR


T. Seat Frame Setup

1. Setting Seat Depth

- Using a 4mm Allen wrench and a 10mm wrench, remove seat pan from frame (See Section CC). If lengthening more than 2" (5.08 cm) for even depths or 1" (2.54 cm) for odd depths, new seat pan parts will be required. Please contact Customer Service for assistance.
- Using a 4mm Allen wrench and a 10mm wrench, remove button head screws (Fig. 43:A, B, and C) from both sides of chair.
- Slide front seat tube in or out as desired. Replace screw B at proper position as shown in the table of Figure 43 for your respective front seat tube. Do not tighten screw/nut.
- Fasten seat frame into yoke at position indicated in table. Replace screw A, but do not tighten screw/nut.
- Fasten mech lock mount into position indicated in table. Replace screw C. Do not tighten screw/nut.
- Repeat for both sides of chair. Once all screws are in place, screws/nuts may be tightened.

⚠ WARNING ⚠

WARNING: The distance from screw A to screw C is ALWAYS to be exactly 9" (22.86 cm). If set to different value, damage to chair can occur.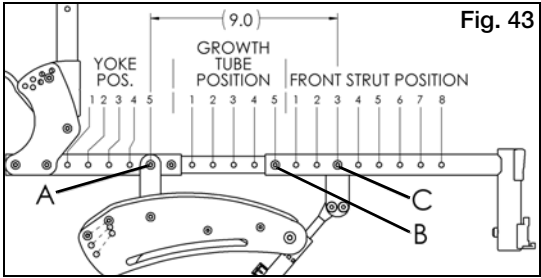

FRONT FRAME SIZE	FRAME DEPTH	YOKE POS.	GROWTH TUBE POS.	FRONT STRUT POS.
SHORT	12	1	1	3
	13	1	2	2
	14	2	3	2
	15	2	4	1
	16	3	5	1
MED	14	2	1	4
	15	2	2	3
	16	3	3	3
	17	3	4	2
	18	4	5	2
LONG	16	3	1	5
	17	3	2	4
	18	4	3	4
	19	4	4	3
	20	5	5	3


V. SET UP & USE OF YOUR WHEELCHAIR

T. Seat Frame Setup

Verify the Validity of the Seat Frame Setup (Fig. 44)

Set the chair with all four wheels on a flat surface and set the seat to a level position, using a bubble level if possible. Dimension Y, which is the vertical distance from the center of the front caster arm pivot bolt to the top of the seat tube, should be between 4 and 5 inches (10.16 - 12.7 cm). If it is less than 4" (10.16 cm) the rear seat height should be raised or the front seat height should be lowered until Dimension Y is between 4 and 5 inches (10.16 - 12.7 cm). If it is more than 5" (12.7 cm), the front seat height should be raised or the rear seat height lowered until Dimension Y is between 4 and 5 inches (10.16 - 12.7 cm).

Optionally, Dimension Y can exceed 5" (12.7 cm), but the posterior tilt range will be reduced accordingly. Ensure casters are square when setup is complete.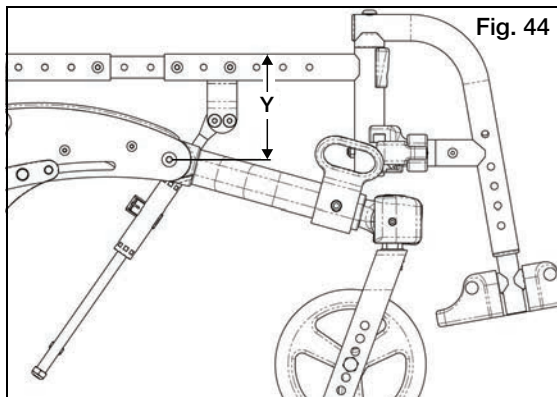


U. Casters

1. Adjusting Caster Arm Position

- Using a 4mm Allen wrench (5mm if transit is installed) and a 10mm wrench, remove screw (Fig. 45:B) and slide caster extension in or out as desired.
- Replace screw and nut and tighten. Repeat for opposite side of chair.

2. Adjusting Caster Arm Angle

- Using a 3mm Allen wrench and a 10mm wrench, remove screw (Fig. 45:A) with mating fastener and rotate caster arm to desired position.
- Replace screw and fastener and tighten. Repeat for opposite side of chair.

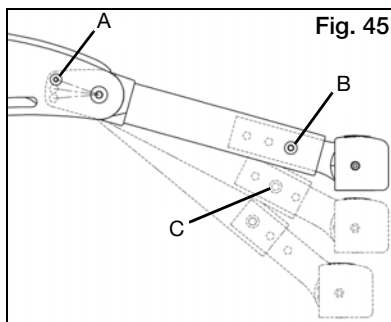NOTE: If desired, height adjustment can be changed with negligible change in wheelbase by extending caster extension while increasing caster arm angle (See Fig. 45:B and C as an example).

DANGER

DANGER: Failure to comply with the instructions above may cause the wheelchair to tip forward during use and result in a fall or loss of control and may cause serious injury or death.


V. SET UP & USE OF YOUR WHEELCHAIR

U. Casters

3. Adjusting Caster Height

- Remove hex bolt (Fig. 46:A) and nut using 13mm wrenches.
- Remove caster wheel (Fig. 46:B) and spacers (Fig. 46:C).
- Feed hex bolt thru desired hole in one side of the caster fork.
- Slide one spacer over hex bolt.
- Slide caster wheel over hex bolt and push hex bolt thru caster wheel, flush with the opposite side of the caster wheel bearing.
- Slide second spacer between caster wheel and caster fork, aligning spacer with bearing hole.
- Push hex bolt thru spacer and hole in opposite side of caster fork.
- Retighten hex bolt with nut.
- Repeat for opposite side of chair. Ensure both casters are set up identically.

Fig. 46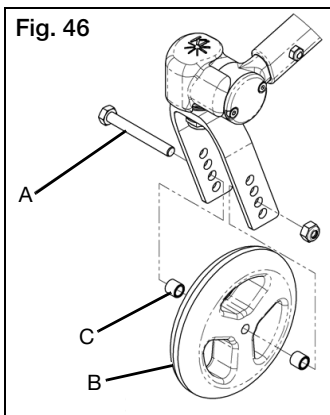

4. Squaring Caster Stem

- Place chair on a flat, hard surface.
- Using 4mm Allen wrench, remove screws (Fig. 47:A) and caster cover plate (Fig. 47:B).
- Loosen center caster barrel screw (Fig. 47:C) approximately 1-2 turns. Do not remove.
- Rotate caster fork outward as shown (Fig. 47).
- Use square to align caster fork with vertical (Fig. 48).
- Thread screws (Fig. 47:A) into the pair of holes that most closely aligns with caster fork square. In Fig. 48, holes "A" are closest. Your setup may use different holes.
- Tighten center screw.
- Remove screws (Fig. 47:A) and replace, capturing cover (Fig. 47:B) underneath screws.
- Repeat for opposite side of chair.

Fig. 47


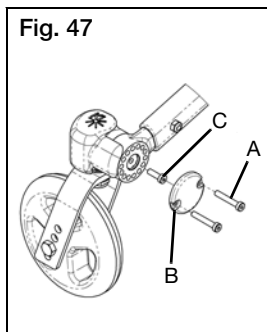
Fig. 48

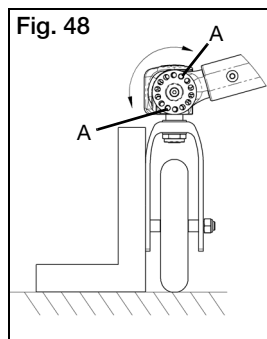V. SET UP & USE OF YOUR WHEELCHAIR

V. Axle Arm



1. Adjusting Axle Arm Angle

- Locate the set screw on each side of the frame (Fig. 49:A).
- Loosen the set screw on each side of the frame using an M3 Allen wrench. Do not remove.
- Adjust footrest tube up or down to achieve the desired height (Fig. 49:B).
- Ensure both sides are adjusted equally.

2. Adjust Axle Tube Position

- Using 24mm wrench, remove axle sleeve nuts (Fig. 50:B), axle sleeve (Fig. 50:A) and washers (Fig. 50:C). Repeat on opposite side of chair.
- Align axle tube with desired hole in axle plate.
- Replace axle sleeves with nuts and washers, ensuring that axle tube position is the same on both sides of chair.
- Repeat for opposite side of chair, ensuring axle bar angle is the same on both sides of the chair.

NOTE: Change in seat height can be achieved with negligible change in wheelbase by simultaneously changing axle arm angle along with axle tube position (See Fig. 49, positions B and C as an example). Similarly, wheelbase can be changed with negligible change in seat height (See Fig. 49, positions C and D as an example).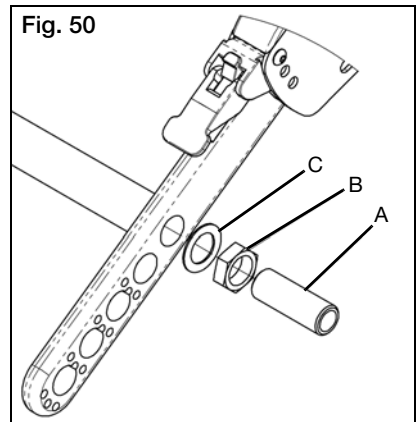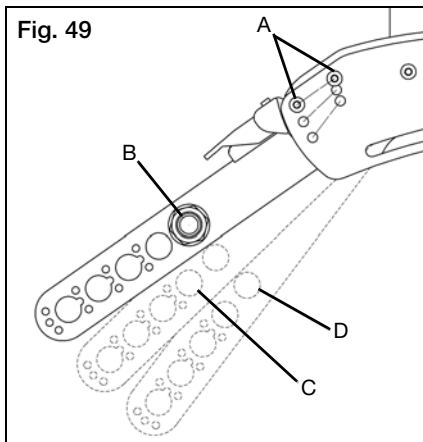


V. SET UP & USE OF YOUR WHEELCHAIR

V. Axle Arm

3. Adjusting Axle Spacing

- Loosen axle sleeve nut (Fig. 51:B) using 24mm wrench.
- Thread axle sleeve in or out of axle tube to achieve desired wheel spacing.
- Retighten axle sleeve nuts.
- Ensure both sides of the chair have the same axle spacing.
- Retighten axle sleeve nuts.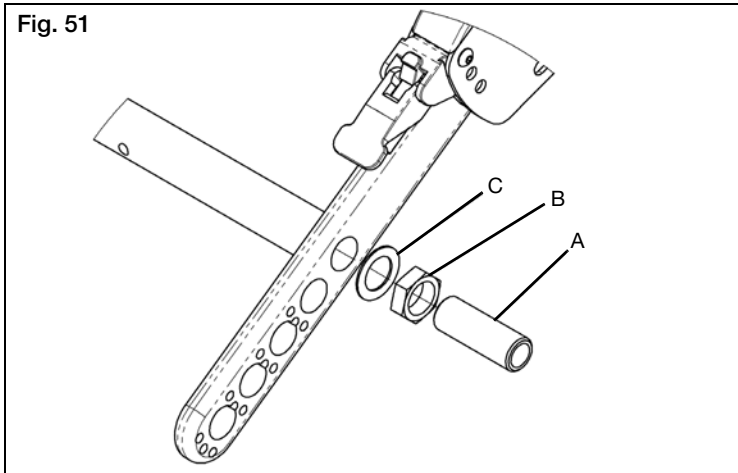

⚠ DANGER ⚠

DANGER: Adjusting the axle position too far forward can result in a chair that may tip over backward resulting in a fall or loss of control that could cause severe injury or death.

⚠ WARNING ⚠

WARNING: Increasing the seat to back angle will change the center of gravity of the wheelchair. Make sure your axle position is adjusted properly in order to maintain constant stability of the wheelchair. Reference instruction for axle arm position in Section V: Axle Arm.


V. SET UP & USE OF YOUR WHEELCHAIR

W. Rear Wheels

1. Installing Wheels

Identify if your chair uses the quick release button on the axle (Fig. 52:B) and if your chair uses the button, continue to next steps. If your chair uses a threaded axle, you will see a bolt head instead of the quick release button. Threaded axle wheels are designed to not be removed so the following steps don't apply.

- a. Push the quick release button (Fig. 52:B) on the axle (Fig. 52:C) to allow the locking balls (Fig. 52:D) to retract. Make note of the difference between the extended and depressed position of the axle release button and its effect on the locking balls on the end of the axle.
- b. Push on quick release button again and slide axle into axle sleeve (Fig. 52:A).
- c. Release the button to lock axle in sleeve. If release button does not fully extend and the locking balls do not move into the locked position after releasing the button, the axle length needs to be adjusted.

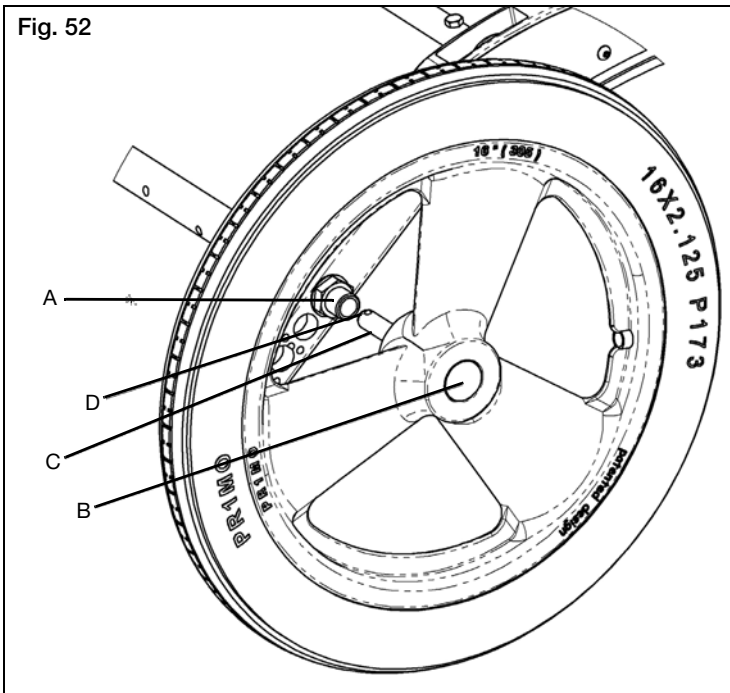V. SET UP & USE OF YOUR WHEELCHAIR

W. Rear Wheels

2. Adjusting Axles

- a. To adjust the axle you will need a 19mm wrench to turn the outside axle nuts.
- b. You will also need a 11mm wrench to hold the ball bearings on the opposite end of the axle and prevent the axle from turning.
- c. When the wheel is installed, if the axle does not lock, loosen the axle nut until it locks and axle can move in and out slightly when locked. If the axle locks, but moves in and out slightly, proceed to next step.
- d. Turn the axle nut counterclockwise until it is tight, then turn the axle nut clockwise 1/3 turn at a time. Check to see if it is properly adjusted after each 1/3 turn. When properly adjusted there should be no perceptible movement of the axle in and out in the sleeve, but the axle release button should be easy to push in.

3. Removing Wheels

- a. Hold the wheel close to the hub and push in the button on the outside end of the axle.
- b. While still holding the button, pull the wheel and axle out of the axle sleeve.

DANGER: Make sure the push button is completely extended and the locking balls on the inside of the chair are fully engaged before operating the wheelchair. Failure to do so may result in the wheel falling off and may cause serious injury or death.

Review and understand Section W. Rear Wheel Installation and Removal before attempting an axle adjustment!


V. SET UP & USE OF YOUR WHEELCHAIR

X. Drum Brake

1. Brake Operation

- a. Squeeze brake levers (Fig. 53:A) on both sides of chair to activate brakes. Each side is operated independently and may be squeezed with more or less force to get more or less braking action.
- b. Let go of brake levers to release brake.
- c. For parking brake function, squeeze each brake lever and, with trigger squeezed, use index finger to flip locking lever (Fig. 53:B) down into teeth on lever mount. While holding locking lever down, release brake handles and parking brake will remain engaged.
- d. To release parking brake, squeeze brake levers until spring loaded locking levers snap back. Release brake levers to deactivate the brake.


Fig. 53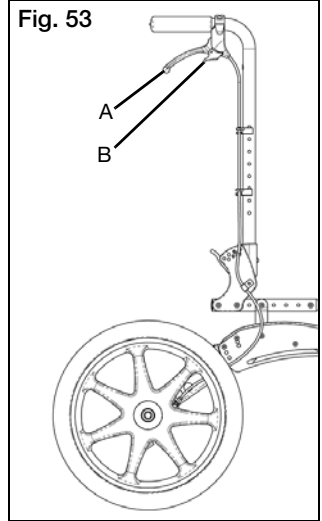

2. Brake Adjustment

- a. Using a 10mm wrench, unlock jam nut (Fig. 54:A) on adjuster (Fig. 54:B). Unthread adjuster to get tighter braking action or thread adjuster in to reduce braking action.
- b. It is recommended to adjust the brake until there is a slight bit of drag on the wheel when spun. Then thread the adjuster inward about $\frac{1}{2}$ turn, or whatever is necessary to remove all drag from the spinning wheel.
- c. Retighten jam nut to lock the adjustment in place. Repeat for opposite side of chair.

Fig. 54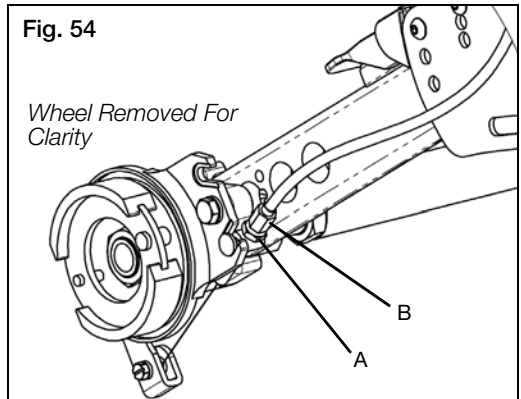


V. SET UP & USE OF YOUR WHEELCHAIR

X. Drum Brake

3. Axle Sleeve Position Adjustment

- a. Remove both wheels. Using a 24mm wrench, loosen jam nut (Fig. 55:A) and unthread all the way onto unthreaded portion of strut adapter (Fig. 55:B). Repeat for opposite side of chair.
- b. Using a 24mm wrench, loosen strut adapter and thread all the way into strut (Fig. 55:C), which will also unthread it from the axle sleeve of the brake assembly. Repeat for opposite side of chair.
- c. Remove washers, move brake assemblies to new position and replace washers.
- d. With brake assemblies seated against the outside of the axle plates, rethread strut adapters onto axle sleeves on each side of the chair. Do not force axle plates outward or inward to begin threads. If there is a gap between the end of the strut adapter and the axle sleeve, thread the adapter outward to meet the sleeve (again, make sure that the brake assembly is seated against the axle plate) - do not force the axle plate inward to start the threads. If there is interference between the adapter and the axle sleeve, thread the adapter into the strut tube further until threads can be started without flexing axle plates outward.
- e. Tighten strut adapters. Strut should be centered between axle plates. Strut can be threaded one way or the other on strut adapters to center the strut after adapters are tightened. Ensure strut is oriented so that foot tilt pedal is approximately horizontal when chair is unfolded.
- f. Strut should neither be forcing axle plates outward nor drawing them inward when tightened. Ensure proper function of axle plate folding.
- g. Cables may need to be adjusted after repositioning axle sleeve.

NOTE: Axle sleeve spacing may not be adjusted with drum brake option.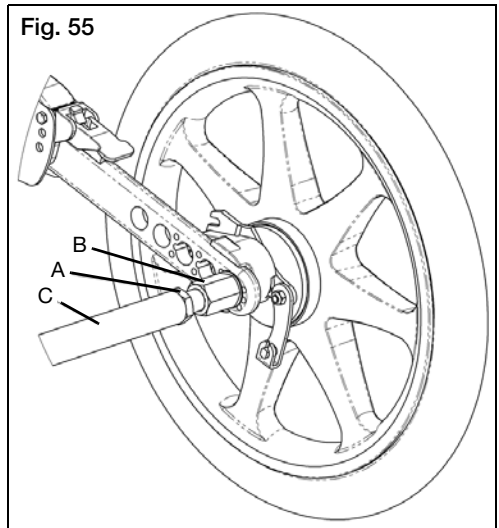

V. SET UP & USE OF YOUR WHEELCHAIR

Y. Push / Pull to Lock Wheel Locks



1. Adjustment

- For standard axle plate, use a 10mm wrench or socket to loosen (do not remove) hex bolt (Fig. 56:F). Self-propel axle plate requires a 5mm Allen wrench to loosen socket head cap screw (Fig. 57).
- Slide mount (Fig. 56:J) forward or rearward to achieve proper locking.
- Screw may be completely removed and slide nut (Fig. 56:G) can be mounted in alternate holes (A & B or B & C) to optimize adjustment range. It is recommended that the threaded hole of slide nut is positioned in either hole A or B.
- Retighten bolt. Repeat for opposite side of chair.

2. Reversing Wheel Lock Mount (Standard Axle Plate Only)

- Using a 10mm wrench or socket, remove hex bolt (Fig. 56:F) completely and remove wheel lock assembly from chair.
- Using a 10mm wrench or socket, remove nuts and washers (Fig. 56:H) from wheel lock studs and remove wheel lock (Fig. 56:I) from mount.
- Flip mount around and reassemble wheel lock to mount. Wheel lock may be installed in either hole D or E to position wheel lock for optimum effectiveness.
- Reinstall wheel lock onto side frame and into slide nut (Fig. 56:G). Note that slide nut can be positioned in holes A & B or B & C to optimize adjustment range. It is recommended that the threaded hole of slide nut be placed in either hole A or B.
- See 1. Adjustment above for wheel lock adjustment procedure.
- Repeat for opposite side of chair. Ensure wheel lock is in the same orientation on both sides of chair.

NOTE: Always loosen and tighten wheel hardware by alternating between the two bolts while loosening/tightening a little at a time. This prevents overclamping on one set of hardware which can lead to binding of the fasteners and increased difficulty in removal.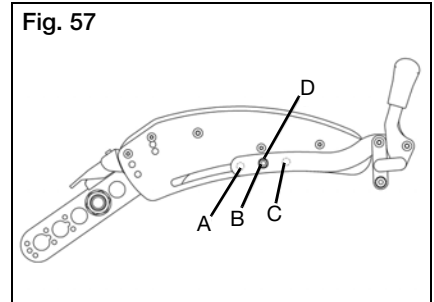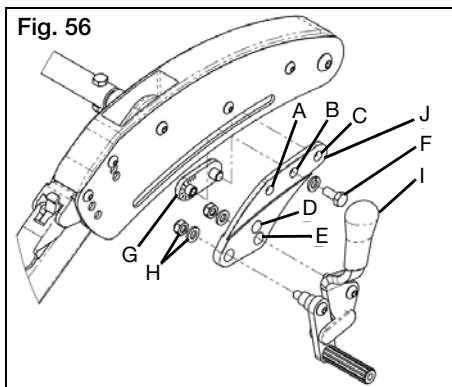


V. SET UP & USE OF YOUR WHEELCHAIR

Z. Attendant Foot Lock (Hub Lock)

- Remove both wheels from wheelchair (see Section W.).
- Using 24mm wrench, loosen (but do not remove) both axle sleeve nuts (Fig. 58:A).
- Twist axle tube to desired angle. If foot tilt is installed, it is not recommended to make large changes to foot pedal as this can over stress the cables attached to the foot tilt.
- Retighten nuts and replace wheels.
- Ensure rear frame still folds properly for storage.

Fig. 58

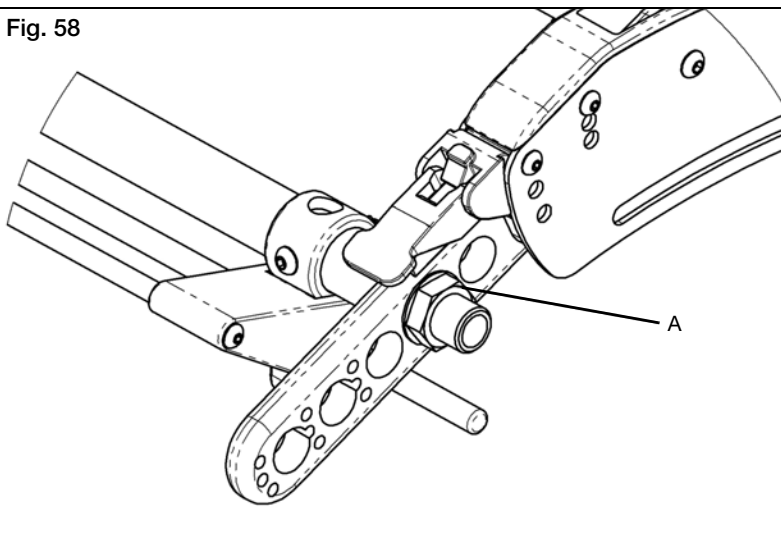
V. SET UP & USE OF YOUR WHEELCHAIR

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1. Frame Width Adjustment

- a. Remove backrest and seating. If a seat pan is installed, a new seat pan will be required for the new seat width. Please contact customer service for assistance.
- b. Remove screws that attach all cross struts - front frame strut (Fig. 59:A), seat frame strut (Fig. 59:B), rear frame strut (Fig. 59:C) and axle strut (Fig. 59:D).
- c. Remove screws for height adjustable back strut. If adjusting from a width of less than 14 to a width of more than 16, new push handles will be required. Please contact customer service for assistance.
- d. Install new struts for new seat width, leaving all nuts and bolts only finger tight. Note that the rear frame strut is identical to the front frame strut, only 3 inches (7.62 cm) shorter.
- e. Place chair on a flat hard surface. With all four wheels contacting the surface, retighten all strut screws.
- f. Verify smooth tilt function.

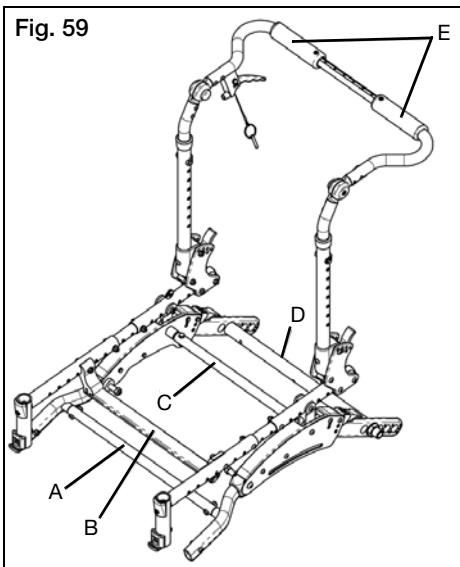
V. SET UP & USE OF YOUR WHEELCHAIR

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2. Attendant Foot Lock Width Adjustment

- a. Remove both wheels. Note how much of the axle sleeve is protruding from nut. Using a 24mm wrench, remove axle sleeve, washer and nut (Fig. 60:A) from axle tube (Fig. 60:B). Repeat for opposite side of chair.
- b. Remove axle tube and lock assembly from chair. Using a 4mm Allen wrench and a 10mm wrench, remove screw (Fig. 60:C) from rod guide (Fig. 60:D). Repeat for opposite side of chair.
- c. Using two 3mm Allen wrenches, while holding screw (Fig. 60:E) on one side of chair, remove corresponding screw on opposite side of chair.
- d. Slide link rod (Fig. 60:F) out from rod levers (Fig. 60:G). Remove other screw from link rod.
- e. Slide rod guide and rod lever outward enough to expose E-Rings (Fig. 60:H). Using a pair of pliers, remove E-Ring (Fig. 60:H) from locking rod (Fig. 60:I). Repeat for opposite side of chair.
- f. Align new locking rod with old locking rod (end to end) and push the old locking rod from the rod guide out with the new, being sure to keep spring inside of rod guide in place, keeping the new rod above the spring. Use a flat screwdriver through end slot if necessary to keep spring compressed while inserting new rod. Continue insertion of new rod through rod guide on opposite side of chair, again making sure that the spring remains beneath the new rod. If spring slips and extends before getting the new rod in place, use a flat screwdriver to retract the spring through the side slots in the rod guide.
- g. Position rod guide and rod lever outboard of the E-Ring grooves in the new locking rod. Reinstall E-Rings and slide rod guides/rod levers in against E-Rings.
- h. Slide new link rod through rod levers and reattach with screws (Fig. 60:E) using two 3mm Allen wrenches.
- i. Align holes in rod guides with cross holes in axle tube and reinstall screws (Fig. 60:C).
- j. After adjusting the width of the rest of the chair (See Section AA.), slide reassembled axle tube assembly between the axle plates and reinstall the axle sleeves with washers and nuts. Ensure that the axle protrudes the same amount on both sides of the chair and that it is at the same distance as it was before removing in step 1.
- k. Reinstall wheels and verify function.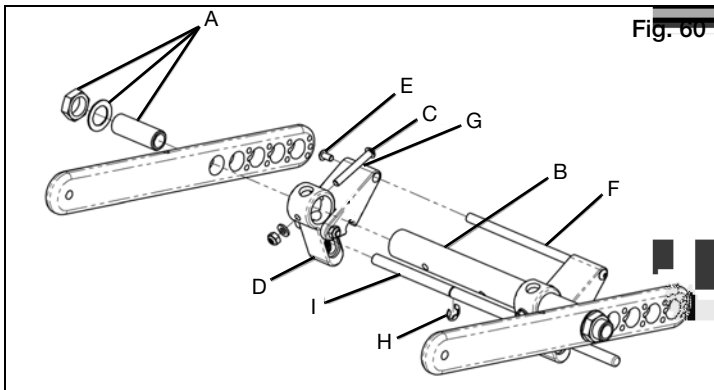

V. SET UP & USE OF YOUR WHEELCHAIR


BB. Anti-Tips

1. Disengaging Anti-tips

- Pull Anti-tip down parallel to the adjustment tube and rotate upward as shown in Fig. 61.
- Anti-tip may be left floating for curb navigation, or it may be rotated all the way up into the detent (Fig. 61:A) to hold it up.
- To re-engage, lift anti-tip back up over detent (if engaged in detent) and push downward to re-latch into place.

2. Removing Anti-tips

- Using a 4mm Allen wrench and a 10mm wrench or socket, remove pivot screw and nut (Fig. 61:B).
- Using a 3mm Allen wrench and an 8mm wrench or socket, remove angle adjustment screw and nut (Fig. 61:C).
- Remove anti-tip from chair along with anti-tip bushing from axle plate hole.
- It is recommended to reassemble screws and nuts with bushing back together to avoid loss of parts until future use.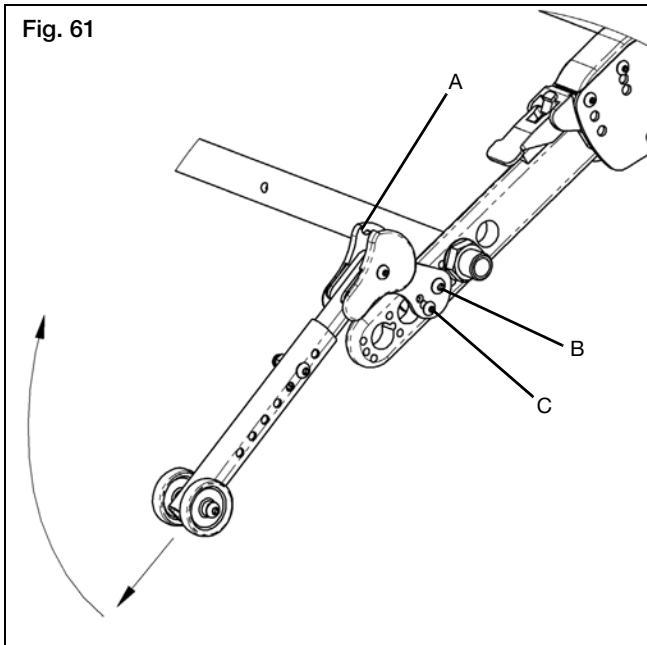

V. SET UP & USE OF YOUR WHEELCHAIR


BB. Anti-Tips

3. Adjusting Anti-tip Angle

- In most cases, the angle of the anti-tip to correlates to the axle plate angle.
- Using a 3mm Allen wrench and an 8mm wrench or socket, remove angle adjustment screw and nut (Fig. 61:C).
- Refer to Fig. 62. When axle bar angle is in position A, hole D in the anti-tip should be aligned with hole G in the axle plate. Note that holes G, H and J are labeled on the endmost hole of the axle plate for clarity, but the corresponding hole under the anti-tip should be used in whatever position the anti-tip is in.
- Similarly, for axle plate position B, hole E in the anti-tip should be aligned with hole H in the axle plate and for axle plate position C, hole F in the anti-tip should be aligned with hole J in the axle plate.
- After retightening the screws and nuts, repeat for opposite side of chair, ensuring both sides are set up identically.

4. Adjusting Anti-tip Height

- Using a 4mm Allen wrench and a 10mm wrench or socket, remove extension screw and nut (Fig. 62:I).
- Align extension tube hole with anti-tip hole so that 1.5" to 2" (2.54 - 5.08 cm) gap is achieved between the floor and the bottom of the anti-tip wheel.
- Retighten extension screw and nut and repeat for opposite side of chair. Ensure that both anti-tips are set up identically.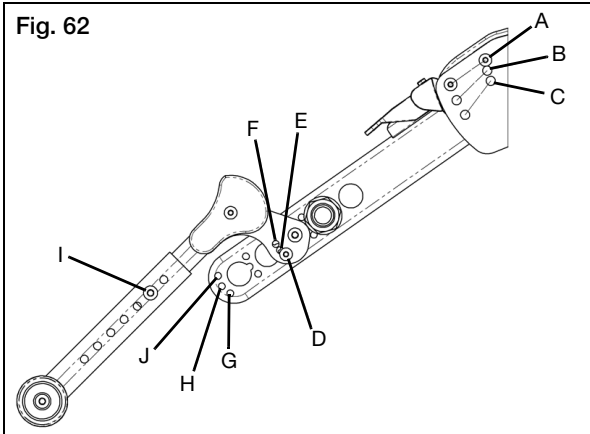

V. SET UP & USE OF YOUR WHEELCHAIR

CC. Seat Pan

1. Removal


- Using a 4mm Allen wrench and a 10mm wrench or socket, remove seat pan bracket screws (Fig. 63:A), washers (Fig. 63:B) and nuts (Fig. 63:C) on all four seat pan brackets (Fig. 63:D).
- Lift entire seat pan from chair.

2. Installation

- Install seat pan brackets (Fig. 63:D) onto front seat pan (Fig. 63:F) using seat pan screws (Fig. 63:E). Use frontmost hole in seat pan. Repeat for opposite side of chair.
- Install seat pan brackets onto rear seat pan (Fig. 63:G) using seat pan screw (Fig. 63:E). Different seat pan holes (Fig. 63:H) may be used in the rear seat pan to avoid interference with the seat frame yoke (Section T. Seat Frame Setup - Fig. 43:A).
- Place seat pans onto seat frame and install carriage bolts (Fig. 63:I) with nuts finger tight.
- Once desired position of seat pan is obtained, tighten carriage bolt nuts.
- Install seat pan bracket screws (Fig. 63:A), washers (Fig. 63:B) and nuts (Fig. 63:C) on all four seat pan brackets.

3. Length Adjustment

- When adjusting frame depth of wheelchair, loosen carriage bolt nuts (Fig. 63:I). Do not remove.
- Slide seat pan forward or rearward corresponding to frame depth and retighten carriage bolt nuts.
- If seat pan length cannot be adjusted to accommodate new seat frame depth, new front and/or rear seat pans will be required. Please contact customer service.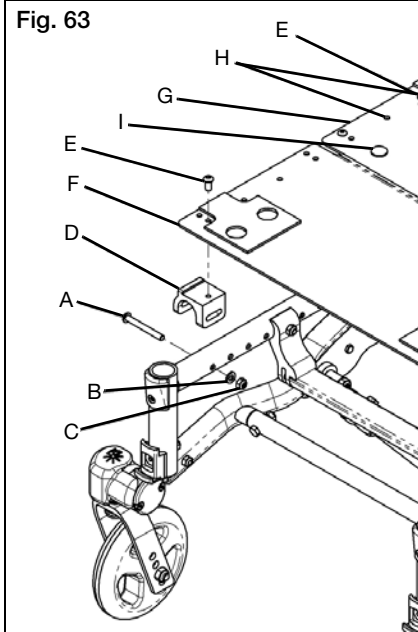


V. SET UP & USE OF YOUR WHEELCHAIR

DD. Bushings

1. Bushing Replacement (It is recommended that this procedure be performed by a qualified technician)

- a. Note which hole the yoke is aligned with in the seat frame. Using a 4mm Allen wrench and a 10mm wrench or socket, remove screws and nuts that hold seat frame to yokes (Fig. 64:E) on both sides of chair and flip seat forward.
- b. Using a 4mm Allen wrench, remove pivot screw (Fig. 64:A) and pivot sleeve (Fig. 64:B). Note that the nut (Fig. 64:C) will fall out of pocket in rear frame strut mount (Fig. 64:D). Keep nut in safe place for future reinstallation.
- c. Carefully slide yoke (Fig. 64:E) up and out of the base frame.
- d. Remove old bushings (Fig. 64:F) from the legs of the yoke. Note that bushings are to be pushed through from the outside of the legs. Ensure all old bushing material is removed and holes are smooth.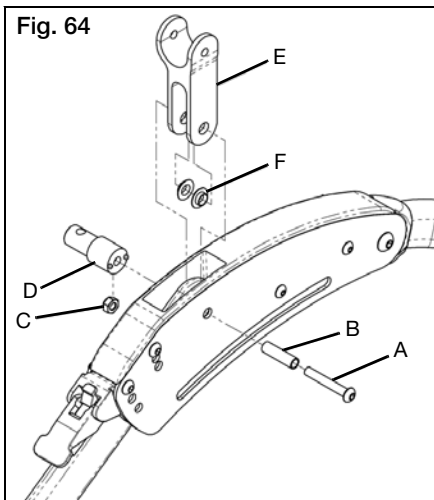

- e. If bushings are worn through, or if there is other damage to the yoke that prevents proper function, replacement yokes are required. Please contact customer service.
- f. Install new bushings from between the legs of the yoke. It may be helpful to push bushings in by using the side of a knife blade or other thin, flat, rigid object, taking care not to damage new bushings. It is not recommended to use fingers as this may skew the bushings and damage them while installing. Ensure bushings are installed straight. Do not use excessive force to get them into place - very little force should be necessary to push bushings in straight. When finished, the flanges of the bushings should be resting flat against the insides of the yoke legs. A replacement yoke is required.
- g. Slide yoke with new bushings in between base frame plates with legs of yoke straddling axle plate. Note that the flatter side of the yoke is to be facing the outside of the chair.
- h. Align holes of base frame plates, yoke and axle bar and reinstall pivot sleeve. It may be helpful to use a long, round object such as a #2 Philips screwdriver to pre-align the holes for easier installation. Do not force sleeve into place as this could damage one or more components, including the bushings.
- i. Slide pivot screw (Fig. 64:A) through sleeve. While holding nut (Fig. 64:C) in pocket of rear frame strut mount (Fig. 64:D), thread screw into nut and tighten, ensuring small pegs on face of strut mount align with holes on the inside base frame plate.
- j. Repeat for opposite side of chair.
- k. Reinstall seat frame in same position as installed previously. The distance between the front strut mount screw (Section T: Fig 43:C) mount and the yoke mount screw (Section T: Fig. 43:A) should be 9" (22.86 cm) (See Section T: Seat Frame Set Up). Check to verify smooth tilt operation.

V. SET UP & USE OF YOUR WHEELCHAIR

EE. Canopy

1. Folding (Fig. 65)


- Close** - Lift the front end of the canopy and take the front end all the way to the back end to fold the canopy out of the way.
- Open** - Take the front end and pull out and down until the canopy is fully stretched out.

2. Quick Removal and Quick Install (Fig. 65)

- Push in the latches on both sides of the canopy and lift up and out of receivers.
- To reinstall canopy, install both ends into the receivers until latches catch the bottoms of the tube mount clamps and "lock" into place.

3. Rain Catch (Fig. 65)

- The edge of the canopy will fold up to catch and direct rain away from the chair user.

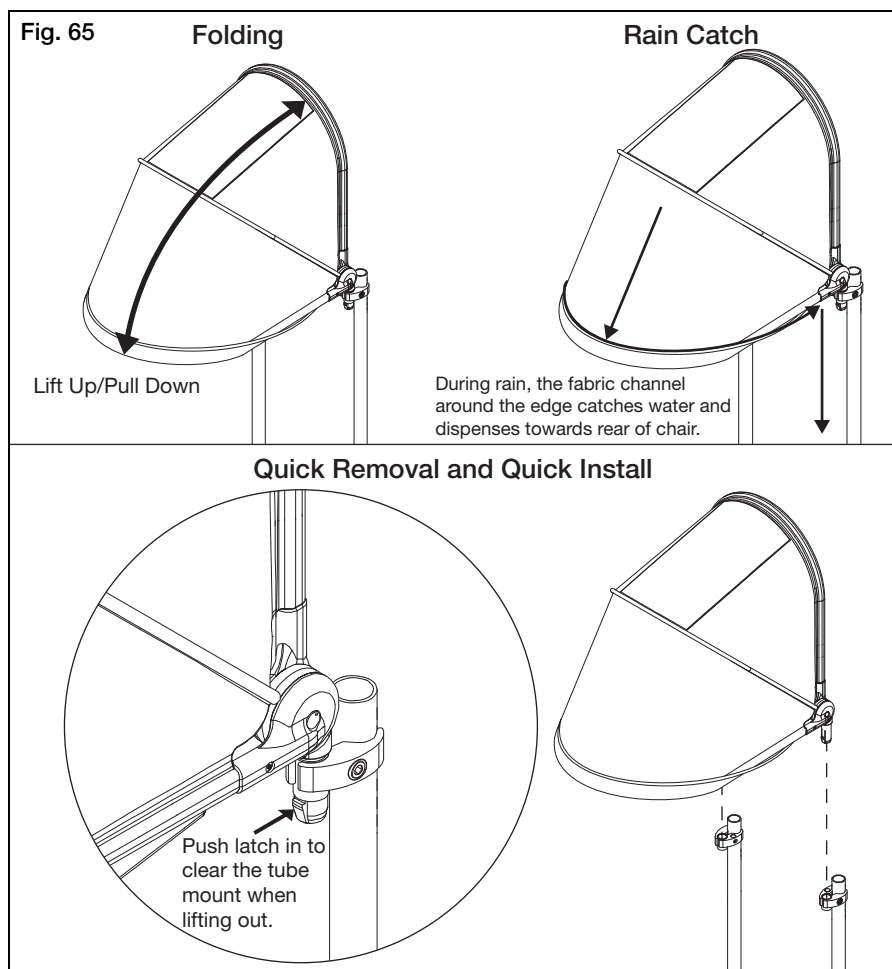VI. MAINTENANCE

A. Inspecting Your Wheelchair

Regular and routine maintenance will extend the life of your wheelchair while improving its performance. Wheelchair repairs and the replacement of parts should be done by a qualified technician or an authorized Ki Mobility supplier.

1. General Inspections:

- a. Clean your chair at least once per month. You may need to clean your chair more frequently if you operate it in dirty environments, such as a worksite.
- b. Check to be sure that all fasteners are tight. Unless otherwise noted, fasteners should be tightened to 40 in./lbs (4.52 N*m).
- c. Check tires and casters:
 - Check the tire for tread wear. Replace the tires if they have flat spots, visible cracks or if the tread is worn off.
 - If you have inflatable tires with a valve stem, check the pressure and set to the pressure listed on the tire sidewall.

DANGER: Replace worn tires. The wheel locks will not grip properly if you fail to maintain the air pressure shown on tire sidewall. This could result in a fall or loss of control and cause severe injury or death.

- d. Check spoke wheels for loose spokes.
- e. Check your wheel locks. As tires wear the wheel locks should be adjusted. See Section X. Drum Brake, Y. Push / Pull Wheel Locks or Section Z. Attendant Foot Lock (Hub Lock).

VI. MAINTENANCE

B. Routine Maintenance

1. Weekly:

- a. Check wheel locks to be sure they are adjusted correctly.
- b. Check axle sleeves to ensure the axle sleeve nuts are tight.
- c. Check for broken, bent or loose spokes.
- d. Check that casters spin freely.
- e. Inspect tires and casters for wear spots.
- f. Check pneumatic tires for proper inflation.
- g. Wipe mech lock rods with a soft cloth.
- h. Adjust tilt cable to ensure proper tilt lock function (See Section Q: Hand Tilt Mechanism or Section R: Foot Tilt Mechanism).
- i. Inspect roller bearings to ensure proper function and wear.
- j. Ensure hand grips do not rotate or pull off.

2. Monthly:

- a. Inspect rear wheel axles and tighten if necessary.
- b. Inspect caster housing bearings for hair build up and remove if necessary.
- c. Inspect wheel locks to be sure assembly is tight. Make sure wheel locks properly engage the tires.
- d. Check that all fasteners are tight and secure.
- e. Inspect frame for any deformities, defects, cracks, dimples or bends. These could be signs of fatigue in the frame which could result in a failure of the chair. Discontinue use of the wheelchair immediately and contact your authorized Ki Mobility dealer.

3. Annually:

- a. Have wheelchair checked and adjusted by a qualified technician.

WARNING

WARNING: After adjustments and before using this wheelchair, ensure all fasteners are tight and secure or injury or damage may occur.

CAUTION

CAUTION: Do not overtighten fasteners as this could damage the frame tubing.

DANGER

DANGER: Failure to read and comply with these instructions may result in a fall or loss of control causing severe injury or death to the user or others.

VI. MAINTENANCE

C. Cleaning

1. Axles and Wheels:

- a. Clean around the axles and wheels WEEKLY with a damp rag.
- b. Hair and lint will lodge in the caster housing. Disassemble the caster housing every six months to remove entangled hair.

NOTE: Do not use WD-40 or any other penetrating oil on this wheelchair. This will destroy the sealed bearings.

NOTE: Do not use any chemical cleaning agents on casters or tires.

WARNING

WARNING: Failure to read and comply with these instructions may result in damage to your wheelchair, a fall or loss of control causing severe injury to the user or others.

D. Storage

1. When not in use, keep your chair in a clean, dry area. Failure to do so may result in your chair rusting and/or corroding.
2. If your chair has been in storage for more than a few weeks you should make sure it is working properly. You should inspect and service, if necessary, all items in Section A. Inspecting Your Wheelchair.
3. If your chair has been in storage for more than two months, it should be serviced and inspected by your authorized supplier before you use it.

WARNING

WARNING: Failure to read and comply with these instructions may result in damage to your wheelchair, a fall, or loss of control causing severe injury to the user or others.

VII. TROUBLESHOOTING

A. Symptoms and Solutions

For optimum performance, your Little Wave "Flip" wheelchair will need to be adjusted occasionally. On the next page is a list of potential symptoms that may occur when adjusting your chair, as well as one or more suggested solutions for each symptom. For best results, when troubleshooting, try one solution at a time before proceeding to a second solution. If symptoms are not relieved, it is recommended to take the chair to an authorized service center for repair.

Please see chart on next page.

Symptoms								Solutions
Chair does not track straight	Loose-ness in Chair	Sluggishness in Turning	One caster does not contact the floor	Caster Flutter	Squeak-ing or rattling	Will not tilt/ Rattling when tilting	Difficulty in tilting	
✓			✓	✓				Add spacer washer between caster stem and bottom caster housing bearing on left or right side of chair.
✓		✓	✓					Ensure all tire pressures are correct and equal on both sides of chair.
✓	✓			✓			✓	Ensure all fasteners are tightened.
					✓			Use Teflon® based lubricant (such as Tri-Flow®) between modular frame components.
						✓	✓	Ensure cable/foot tilt mechanism is adjusted (see section V.K.2 or V.L.2).

VIII. WARRANTY

A. Warranty Information

Ki Mobility warrants the frame and quick-release axles of this wheelchair against defects in materials and workmanship for the life of the original purchaser. All other Ki Mobility-made parts and components of this wheelchair are warranted against defects in materials and workmanship for one year from the date of first consumer purchase.

The expected life of the frame is five years.

Limitations to the Warranty

1. We do not warrant:
 - a. Wear items: Upholstery, tires, armrest pads, tubes, armrests and push-handle grips.
 - b. Damage resulting from neglect, misuse or from improper installation or repair.
 - c. Damage from exceeding weight limit.
2. This warranty is VOID if the original chair serial number tag is removed or altered.
3. This warranty is VOID if the original chair has been modified from its original condition and it is determined the modification resulted in failure.
4. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.

Ki Mobility's Responsibility

Ki Mobility's only liability is to replace or repair, at our discretion, the covered parts. There are no other remedies, expressed or implied.

Your Responsibility

- a. Notify Ki Mobility, via an authorized supplier, prior to the end of the warranty period and get a return authorization (RA) for the return or repair of the covered parts.
- b. Have the supplier send the authorized return, freight pre-paid, to:

Ki Mobility
5201 Woodward Drive
Stevens Point, WI 54481

- c. Pay any charges for labor to repair or install parts.

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