



Activity Chair

In a few short years the Activity Chair has become the classroom standard for children and young adults with special positioning needs. Not only that, but parents who see what these chairs can do in school are getting them for use at home. We've also seen a growing number of speech and feeding specialists adopt this chair in their hospital-based clinics. It's safe to say the Activity Chair has truly revolutionized adaptive seating and continues to delight therapists with its versatility for people of all shapes and sizes, all conditions and special needs.



Find inservice videos, letters of medical necessity and informative articles at:
www.rifton.com/activitychair

"Your Activity Chair provides incredible ease of use with all the durability I've come to trust from Rifton. And all the positioning components meet the demands of even the most involved diagnoses. A great chair. *"*

*Jeremy Elliott, ATP/CRTS
Cumberland, Maryland*

Hi/lo Activity Chair (R870) with
contoured headrest, laterals and tray.

DESIGNED FOR USE WITH THE
MOVE
PROGRAM

Activity Chair features



Spring option
The springs on the standard base (in both backrest and seat) allow for self-generated, dynamic motion—ideal for clients with autism. The spring column can be locked when not in use.



Caster swivel lock
The swivel lock prevents the chair from drifting sideways when it is pushed for longer distances.

Standard base



Adjustable back



Tilt-in-space



Hi/lo base



Adjust while in use

The Activity Chair can be easily adjusted without tools, with a client in the chair. This saves time and makes adjustments much simpler to gauge.



Adjustable back



Tilt-in-space

Easy height adjustment

The large and medium hi/lo bases are height adjustable with the foot pump.



Footboard features



Perfect for the MOVE™ program



When used with forearm prompts, the Activity Chair becomes a forward-leaning instructional seating device as used in the MOVE™ program.

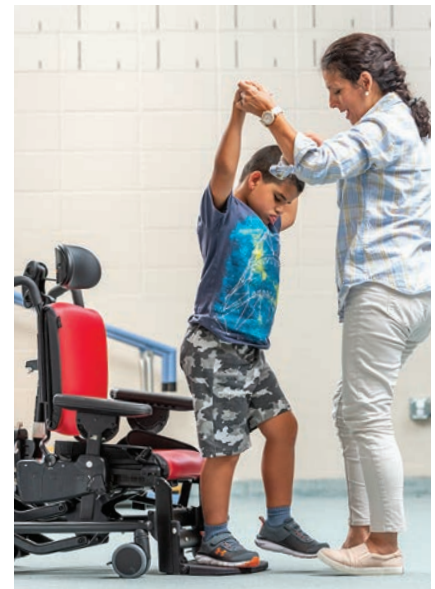
Transfers made easy



Remove the armrests and raise the hi/lo chair to the ideal transfer height for secure, safe lateral moves.



Flip the footboard up and tilt the chair forward for easy sit-to-stand transfers.



Lower the hi/lo chair until the footboard is flat on the floor. Because the footboard is strong enough for stand and pivot transfers, clients can assist in their own transfers.

A chair for everything

The Activity Chair's tilt-in-space feature allows easy transitions between forward leaning for task participation and unimpeded interaction, and reclining for rest or a higher level of support.



All Rifton Activity Chairs have tilt-in-space.

Whether you choose the standard or hi/lo base, you'll get Rifton's quick-adjusting, tilt-in-space feature. The standard base tilts 15° forward and 15° back, while the hi/lo tilts 15° forward and 25° back. Tilt-in-space is a valuable feature because it encourages sit-to-stand and offers a variety of resting positions. Best of all, our tilt-in-space can be activated while your client is in the chair.





This medium standard chair is in posterior tilt to provide support for a student with poor head control.



The tilt-in-space feature accommodates users who need a higher degree of head support.

Infinitely adjustable for every need

The favorite of feeding clinics

Rifton's Activity Chair can be either a basic chair for a client who needs just a little support, or it can offer significant support for more involved individuals. The hi/lo base allows the client to be fed in a quiet, individual setting or at the family dinner table. Because you can measure the angle of the tilt-in-space and backrest recline, home caregivers can exactly duplicate the angles recommended by therapists.

Each size of Activity Chair has so much growth potential that it will provide many years of service. And it is easy to clean—essential for a feeding chair. The tray clips on and off with one hand.

Quickly adjustable tilt-in-space during feeding is beneficial. Therapists may start out the session doing oral motor exercise with the client upright and then tilt back to reduce the need for trunk control. They can then adjust the headrest forward to keep the head in a neutral position.

All these features make it the ideal feeding chair.



From an activity at counter height to a low-table group activity, the hi/lo base chair makes participation easy.

Components



Choice of arm supports

Armrests provide lateral boundaries as well as upper extremity weight-bearing assist to aid trunk control. Rifton's armrests can be raised or lowered individually and can be set at angles from horizontal to 30° up or down. The tray attaches to the armrests.

small 2¼" x 8¾", **large** 3" x 12"



Forearm prompts facilitate head and trunk control while sitting, and secure distal placement of upper extremities. This enables the use of the pelvic girdle and trunk core muscles for improved postural stabilization and strengthening. Rifton's forearm prompts are fully adjustable for optimal positioning.

Padded area: small 4" x 7", large 4½" x 8"

Range of adjustment from elbow to fist: small 8" – 12½", large 10½" – 16"



Choice of adjustable legs

Adjustable legs (long and short): Height-adjustable in 1" increments, these legs allow you to position your client at optimal height. The grippy tips of the stationary legs keep the chair from sliding and moving across the floor, even with dynamic rocking.

Adjustable legs with casters (long and short): Height-adjustable in 1" increments, these legs allow you to position your client at optimal height. The casters make your standard base chair easy to move when transitioning from one classroom to another.



Pads: Comfortable seat and backrest pads are essential for providing cushioned support and preventing pressure sores. The backrest and seat pads are made of polyurethane foam covered with a cleanable, healthcare-grade polyurethane-coated fabric. The seat pad can be turned front to back for longer wear. Pads are available in 6 colors.



Components (continued)



Headrests optimally position clients with poor head control. Choose a flat, contoured or winged headrest. The winged headrest adjusts from almost flat to 90°. All headrests adjust front to back and up and down using Rifton's unique two-knob adjustment system.

Flat and contoured: 9" x 5"

Winged: 16" x 5" when flat to 6" between wings at 90°



Laterals: Necessary for clients who need good upper body support, laterals are important for proper alignment of the spine. Laterals have independent vertical, lateral and angle adjustment.

Small 5" x 4", **large** 6½" x 5"



Laterals with chest strap: With the addition of a wide chest strap to the laterals, clients with limited upper torso control gain added trunk support and security.



Push handles: Recommended for all chairs with wheels, push handles make it easy for a caregiver to maneuver and transport a client in the Activity Chair from one place to another.



Tray: The tray attaches to the armrests and provides a work, play and feeding surface with raised edges. It can be set at angles ranging from horizontal to 30° up or down, and can be raised or lowered to meet your client's needs.

Small 19" wide x 18" deep, **medium** 22" wide x 20" deep

Large 25½" wide x 22½" deep



Handhold: The handhold attaches to the tray for arm and hand positioning. Handholds are especially useful for clients with spastic muscle tone and are helpful for proper joint alignment.





Chest strap: The wide and comfortably padded chest strap provides added trunk support.

Max. circumference: small 34", medium 40", large 52"

Width of strap: small 3", medium 3¾", large 4"



Butterfly harness: When adjusted to fit snugly and comfortably, the butterfly harness provides maximum anterior support giving security, safety and freedom of movement.

Slim-cut butterfly harness performs the same positioning function as the regular butterfly harness. The slim cut butterfly harness is an alternative especially designed for larger female clients.



Thigh belt: The thigh belt provides knee adduction and additional support and security for the client's thighs.

Max. circumference: small 32", medium 41", large 52"



Pelvic harness: An alternative to a typical seat belt, Rifton's unique pelvic harness firmly positions a client's pelvis by securing hips and upper thighs without pressure on the abdomen. The pelvic harness provides a stable, comfortable base for postural control and encourages proper alignment of the spine. It also controls extreme extensor spasticity that can result in sacral sitting.

Max. thigh circumference: small 22", medium 25", large 31"



Hip guides attach to the arm supports. They provide a snug, comfortable fit giving the client security and safety. Placed at the sides of the pelvis to align a client's torso, they provide maximum support of trunk control while sitting. Hip guides can also be important for proper alignment of the spine. They independently adjust vertically and horizontally. For transfers, hip guides remove with the arm supports.

Dimensions: small 3¼" x 5½", large 4" x 9¼"



Components (continued)



Abductor: The abductor separates the knees and helps to control spastic muscles that cause leg scissoring. The abductor can be adjusted forward and back in three positions to improve postural control.

Dimensions: small 3"x 6"x 3½", large 4"x 7½"x 4"



Adductors: Adductors provide a comfortable lateral boundary for client's knees and maintain proper body and joint alignment.

One size only: 5" high x 6" long



Leg prompts are helpful for clients with limited muscle control. They maintain body and joint alignment when sitting by supporting each leg individually with a strap that passes around the front. Leg prompts can be adjusted in and out and laterally.

Max. circumference: small 14", medium 16", large 18"



Footboard: The footboard facilitates weight-bearing through the legs and feet during functional sitting activities, and can be used with sandals or ankle straps to meet specific positioning needs. The footboard can flip up, swing back out of the way for sit-to-stand transfers, or can be used as a step for transfers.



Ankle straps (require footboard) are helpful for clients with limited muscle control and excursion movement. They allow a moderate range of movement while improving independence in postural control.

Note: Ankle straps cannot be used together with sandals.

Max. circumference: small 12", large 16½"



Sandals (require footboard): For clients who are unable to control leg movement, sandals adjust to position client's feet correctly for optimal body alignment.

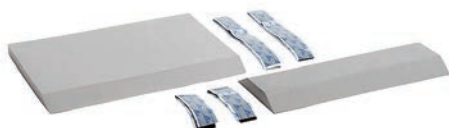
Small 3"x 7", **medium** 4"x 9", **large** 5"x 10½"





Wedges (require sandals) can accommodate clients with specific ankle positioning needs or uneven leg lengths. Use them to raise heels or toes, or combine two wedges under one sandal to add flat lift.

Thickness: small 1", medium 1¼", large 1½"



Lumbar and seat support kit: This kit lets you adapt the contours of the seat and back for a specific client's needs. The foam inserts can be cut to length and attached where needed under the seat and back pads with adhesive-backed hook and loop strips.



Backrest filler pad fills the space between the bottom edge of the backrest and the top edge of the seat when backrest is in one of the top three positions. It provides additional lower back support.



Mini kit: Consisting of a backrest insert pad and a mini trunk support, the mini kit supports children as short as 28" tall. It allows you to begin positioning children correctly at an earlier age, and continue to use the small chair for years before it's outgrown.

Dimensions: Trunk support max. circumference is 26", backrest pad is 1" thick



Whitmyer adapter: Some Whitmyer headrests can be installed on all Rifton Activity Chairs. This adapter gives you the option to use a Whitmyer M2100 headrest mounting bracket and the Whitmyer Onyx headrest support system.

Note: Rifton supplies only the adapter and T20 power tip. Whitmyer mounting brackets and headrests must be purchased from a Whitmyer supplier.



Room to grow

Small chair with mini kit at its smallest dimensions

- 1 Backrest height 12½"
- 2 Trunk support width 5½"
- 3 Seat depth 7"
- 4 Seat width 7"
- 5 Armrest height above seat 5"
- 6 Seat height above footboard 7"

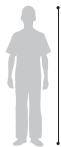


Small chair at its largest dimensions

- 1 Backrest height 15½"
- 2 Distance between laterals 11½"
- 3 Seat depth 12"
- 4 Seat width 12"
- 5 Armrest height above seat 7½"
- 6 Seat height above footboard 12"




Small standard base
R820 Rifton Activity Chair

| User dimensions (inches) | |
|---|---|
| Height with mini kit | 32–48 28–38 |
|  Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth. Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended. | |
| Item dimensions (inches) | |
| Frame width | R820 small Standard base |
| | short legs: 21 long legs: 23 |
| | short legs w/ casters: 22½ long legs w/ casters: 23½ |
| | Seat height above floor |
| | short legs: 9½–12½ long legs: 18½–21½ |
| | short legs w/ casters: 13½–16½ long legs w/ casters: 18½–21½ |
| Seat angle (tilt-in-space)—degrees | 15° forward, 15° back |
| Backrest angle—degrees | 5° forward, 20° back |
| Footboard knee angle—degrees | 45°–110° |
| Seat height above footboard | 7–12 |
| Seat width with hip guides | 7–9 |
| without hip guides | 12 |
| Seat depth | 8–12 |
| with mini kit | 7–11 |
| Armrest height above seat | 5–7½ |
| Trunk support width | 5½–11½ |
| Backrest height | 12½–15½ |
| Headrest height above seat | 14½–21 |
| Max. working load (lb) | 75 |



Small hi/lo base
R830 Rifton Activity Chair

| User dimensions (inches) | |
|---|-----------------------|
| Height with mini kit | 32–48 28–38 |
|  Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth. Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended. | |
| Item dimensions (inches) | |
| Frame width | R830 small Hi/lo base |
| Seat height above floor | 26 10–25 |
| Seat angle (tilt-in-space)—degrees | 15° forward, 25° back |
| Backrest angle—degrees | 5° forward, 20° back |
| Footboard knee angle—degrees | 45°–110° |
| Seat height above footboard | 7–12 |
| Seat width with hip guides | 7–9 |
| without hip guides | 12 |
| Seat depth | 8–12 |
| with mini kit | 7–11 |
| Armrest height above seat | 5–7½ |
| Trunk support width | 5½–11½ |
| Backrest height | 12½–15½ |
| Headrest height above seat | 14½–21 |
| Max. working load (lb) | 75 |

2021 order form for small

Use dimension chart to select appropriate size.



| | | Standard R820 small | | Hi/lo R830 small | | |
|--|--------|---------------------------|--------------------------------|--------------------------------|--------------------------------|----------------------------------|
| * Seat and back includes seatbelt (Hi/lo includes footboard and push handles) | CHOOSE | Adjustable | R821 | \$565 <input type="checkbox"/> | R831 | \$1,155 <input type="checkbox"/> |
| | | Adjustable w/ back spring | R822 | \$645 <input type="checkbox"/> | R832 | \$1,260 <input type="checkbox"/> |
| * Base | CHOOSE | Without spring | R823 | \$330 <input type="checkbox"/> | R833 | \$1,975 <input type="checkbox"/> |
| | | With spring | R824 | \$390 <input type="checkbox"/> | N/A | |
| * Arm supports (pair) | CHOOSE | Armrests | R827 | \$230 <input type="checkbox"/> | R827 | \$230 <input type="checkbox"/> |
| | | Forearm prompts | R809 | \$565 <input type="checkbox"/> | R809 | \$565 <input type="checkbox"/> |
| * Adjustable legs | CHOOSE | Short | R803 | \$45 <input type="checkbox"/> | N/A | |
| | | Long | R894 | \$75 <input type="checkbox"/> | N/A | |
| | | Short with casters | R805 | \$280 <input type="checkbox"/> | N/A | |
| | | Long with casters | R896 | \$310 <input type="checkbox"/> | N/A | |
| * Pads | CHOOSE | Tan | R815 | \$230 <input type="checkbox"/> | R815 | \$230 <input type="checkbox"/> |
| | | Blue | R816 | \$230 <input type="checkbox"/> | R816 | \$230 <input type="checkbox"/> |
| | | Red | R817 | \$230 <input type="checkbox"/> | R817 | \$230 <input type="checkbox"/> |
| | | Pink | R971 | \$230 <input type="checkbox"/> | R971 | \$230 <input type="checkbox"/> |
| | | Purple | R972 | \$230 <input type="checkbox"/> | R972 | \$230 <input type="checkbox"/> |
| | | Green | R973 | \$230 <input type="checkbox"/> | R973 | \$230 <input type="checkbox"/> |
| Headrest | CHOOSE | Contoured | R897 | \$230 <input type="checkbox"/> | R897 | \$230 <input type="checkbox"/> |
| | | Flat | R898 | \$215 <input type="checkbox"/> | R898 | \$215 <input type="checkbox"/> |
| | | Adjustable winged | R881 | \$285 <input type="checkbox"/> | R881 | \$285 <input type="checkbox"/> |
| Laterals (pair) | CHOOSE | Small | R828 | \$190 <input type="checkbox"/> | R828 | \$190 <input type="checkbox"/> |
| | | Small with strap | R819 | \$230 <input type="checkbox"/> | R819 | \$230 <input type="checkbox"/> |
| Push handles (recommended for all chairs with wheels) | | R878 | \$145 <input type="checkbox"/> | Included | | |
| Tray (requires armrests) | | R667 | \$230 <input type="checkbox"/> | R667 | \$230 <input type="checkbox"/> | |
| Handhold | | R649 | \$25 <input type="checkbox"/> | R649 | \$25 <input type="checkbox"/> | |
| Chest strap (wide) | | R802 | \$80 <input type="checkbox"/> | R802 | \$80 <input type="checkbox"/> | |
| Butterfly harness | | R834 | \$110 <input type="checkbox"/> | R834 | \$110 <input type="checkbox"/> | |
| Thigh belt | | R811 | \$80 <input type="checkbox"/> | R811 | \$80 <input type="checkbox"/> | |
| Pelvic harness | | R845 | \$145 <input type="checkbox"/> | R845 | \$145 <input type="checkbox"/> | |
| Hip guides (pair) | | R814 | \$195 <input type="checkbox"/> | R814 | \$195 <input type="checkbox"/> | |
| Abductor | | R884 | \$120 <input type="checkbox"/> | R884 | \$120 <input type="checkbox"/> | |
| Adductors (pair) | | R892 | \$190 <input type="checkbox"/> | R892 | \$190 <input type="checkbox"/> | |
| Leg prompt | | R854 | \$205 <input type="checkbox"/> | R854 | \$205 <input type="checkbox"/> | |
| Footboard (recommended for chairs with long legs) | | R847 | \$445 <input type="checkbox"/> | Included | | |
| Mini kit (backrest insert and trunk support) | | R812 | \$120 <input type="checkbox"/> | R812 | \$120 <input type="checkbox"/> | |
| Ankle straps (pair) | | R899 | \$85 <input type="checkbox"/> | R899 | \$85 <input type="checkbox"/> | |
| Sandals (pair) | | R944 | \$85 <input type="checkbox"/> | R944 | \$85 <input type="checkbox"/> | |
| Wedges (pair, require sandals) | | R697 | \$65 <input type="checkbox"/> | R697 | \$65 <input type="checkbox"/> | |
| Lumbar and seat support kit | | R886 | \$65 <input type="checkbox"/> | R886 | \$65 <input type="checkbox"/> | |
| Backrest filler pad | | R849 | \$120 <input type="checkbox"/> | R849 | \$120 <input type="checkbox"/> | |
| Whitmyer headrest adapter | | R804 | \$55 <input type="checkbox"/> | R804 | \$55 <input type="checkbox"/> | |
| Total | | R820 | \$ _____ | R830 | \$ _____ | |

* Required

Room to grow

Medium chair at its smallest dimensions

- 1 Backrest height 15½"
- 2 Distance between laterals 6½"
- 3 Seat depth 11"
- 4 Seat width 8½"
- 5 Armrest height above seat 7"
- 6 Seat height above footboard 7¾"

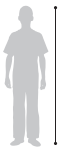


Medium chair at its largest dimensions

- 1 Backrest height 19"
- 2 Distance between laterals 12"
- 3 Seat depth 16"
- 4 Seat width 14"
- 5 Armrest height above seat 10½"
- 6 Seat height above footboard 15¼"




Medium standard base
R840 Rifton Activity Chair

| User dimensions (inches) | |
|---|---|
| Height | 40–56 |
|  Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth. Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended. | |
| Item dimensions (inches) | R840 medium Standard base |
| Frame width | short legs: 23¾ long legs: 24½ |
| | short legs w/ casters: 24½ long legs w/ casters: 25½ |
| Seat height above floor | short legs: 12½–16½ long legs: 19–23 |
| | short legs w/ casters: 16–20 long legs w/ casters: 19–23 |
| Seat angle (tilt-in-space)–degrees | 15° forward, 15° back |
| Backrest angle–degrees | 5° forward, 20° back |
| Footboard knee angle–degrees | 45°–110° |
| Seat height above footboard | 7¾–15¼ |
| Seat width with hip guides without hip guides | 8½–11½ 14 |
| Seat depth | 11–16 |
| Armrest height above seat | 7–10½ |
| Distance between laterals | 6½–12 |
| Backrest height | 15½–19 |
| Headrest height above seat | 17½–24½ |
| Max. working load (lb) | 150 |



Medium hi/lo base
R850 Rifton Activity Chair

| User dimensions (inches) | |
|---|------------------------|
| Height | 40–56 |
|  Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth. Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended. | |
| Item dimensions (inches) | R850 medium Hi/lo base |
| Frame width | 27½ |
| Seat height above floor | 12–23 |
| Seat angle (tilt-in-space)–degrees | 15° forward, 25° back |
| Backrest angle–degrees | 5° forward, 20° back |
| Footboard knee angle–degrees | 45°–110° |
| Seat height above footboard | 7¾–15¼ |
| Seat width with hip guides without hip guides | 8½–11½ 14 |
| Seat depth | 11–16 |
| Armrest height above seat | 7–10½ |
| Distance between laterals | 6½–12 |
| Backrest height | 15½–19 |
| Headrest height above seat | 17½–24½ |
| Max. working load (lb) | 150 |

2021 order form for medium

Use dimension chart to select appropriate size.



| | | | Standard R840 medium | | Hi/lo R850 medium | |
|--|--------|---------------------------|-------------------------|--------------------------------|----------------------|----------------------------------|
| * Seat and back includes seatbelt (Hi/lo includes footboard and push handles) | CHOOSE | Adjustable | R841 | \$580 <input type="checkbox"/> | R851 | \$1,210 <input type="checkbox"/> |
| | | Adjustable w/ back spring | R842 | \$660 <input type="checkbox"/> | R852 | \$1,315 <input type="checkbox"/> |
| * Base | CHOOSE | Without spring | R843 | \$335 <input type="checkbox"/> | R853 | \$2,430 <input type="checkbox"/> |
| | | With spring | R844 | \$405 <input type="checkbox"/> | N/A | |
| * Arm supports (pair) | CHOOSE | Armrests | R868 | \$235 <input type="checkbox"/> | R868 | \$235 <input type="checkbox"/> |
| | | Forearm prompts | R869 | \$590 <input type="checkbox"/> | R869 | \$590 <input type="checkbox"/> |
| * Adjustable legs | CHOOSE | Short | R893 | \$55 <input type="checkbox"/> | N/A | |
| | | Long | R894 | \$75 <input type="checkbox"/> | N/A | |
| | | Short with casters | R895 | \$285 <input type="checkbox"/> | N/A | |
| | | Long with casters | R896 | \$310 <input type="checkbox"/> | N/A | |
| * Pads | CHOOSE | Tan | R835 | \$235 <input type="checkbox"/> | R835 | \$235 <input type="checkbox"/> |
| | | Blue | R836 | \$235 <input type="checkbox"/> | R836 | \$235 <input type="checkbox"/> |
| | | Red | R837 | \$235 <input type="checkbox"/> | R837 | \$235 <input type="checkbox"/> |
| | | Pink | R974 | \$235 <input type="checkbox"/> | R974 | \$235 <input type="checkbox"/> |
| | | Purple | R975 | \$235 <input type="checkbox"/> | R975 | \$235 <input type="checkbox"/> |
| | | Green | R976 | \$235 <input type="checkbox"/> | R976 | \$235 <input type="checkbox"/> |
| Headrest | CHOOSE | Contoured | R897 | \$230 <input type="checkbox"/> | R897 | \$230 <input type="checkbox"/> |
| | | Flat | R898 | \$215 <input type="checkbox"/> | R898 | \$215 <input type="checkbox"/> |
| | | Adjustable winged | R881 | \$285 <input type="checkbox"/> | R881 | \$285 <input type="checkbox"/> |
| Laterals (pair) | CHOOSE | Small | R828 | \$190 <input type="checkbox"/> | R828 | \$190 <input type="checkbox"/> |
| | | Small with strap | R829 | \$235 <input type="checkbox"/> | R829 | \$235 <input type="checkbox"/> |
| | | Large | R838 | \$205 <input type="checkbox"/> | R838 | \$205 <input type="checkbox"/> |
| Push handles (recommended for all chairs with wheels) | | | R878 | \$145 <input type="checkbox"/> | Included | |
| Tray (requires armrests) | | | R668 | \$265 <input type="checkbox"/> | R668 | \$265 <input type="checkbox"/> |
| Handhold | | | R649 | \$25 <input type="checkbox"/> | R649 | \$25 <input type="checkbox"/> |
| Chest strap (wide) | | | R863 | \$85 <input type="checkbox"/> | R863 | \$85 <input type="checkbox"/> |
| Butterfly harness | CHOOSE | Regular | R864 | \$120 <input type="checkbox"/> | R864 | \$120 <input type="checkbox"/> |
| | | Slim cut | R882 | \$120 <input type="checkbox"/> | R882 | \$120 <input type="checkbox"/> |
| Thigh belt | | | R865 | \$85 <input type="checkbox"/> | R865 | \$85 <input type="checkbox"/> |
| Pelvic harness | | | R866 | \$150 <input type="checkbox"/> | R866 | \$150 <input type="checkbox"/> |
| Hip guides (pair) | | | R867 | \$200 <input type="checkbox"/> | R867 | \$200 <input type="checkbox"/> |
| Abductor | CHOOSE | Small | R884 | \$120 <input type="checkbox"/> | R884 | \$120 <input type="checkbox"/> |
| | | Large | R891 | \$140 <input type="checkbox"/> | R891 | \$140 <input type="checkbox"/> |
| Adductors (pair) | | | R892 | \$190 <input type="checkbox"/> | R892 | \$190 <input type="checkbox"/> |
| Leg prompt | | | R858 | \$215 <input type="checkbox"/> | R858 | \$215 <input type="checkbox"/> |
| Footboard (recommended for chairs with long legs) | | | R848 | \$455 <input type="checkbox"/> | Included | |
| Ankle straps (pair) | | | R899 | \$85 <input type="checkbox"/> | R899 | \$85 <input type="checkbox"/> |
| Sandals (pair) | | | R954 | \$90 <input type="checkbox"/> | R954 | \$90 <input type="checkbox"/> |
| Wedges (pair, require sandals) | | | R698 | \$65 <input type="checkbox"/> | R698 | \$65 <input type="checkbox"/> |
| Lumbar and seat support kit | | | R887 | \$75 <input type="checkbox"/> | R887 | \$75 <input type="checkbox"/> |
| Backrest filler pad | | | R849 | \$120 <input type="checkbox"/> | R849 | \$120 <input type="checkbox"/> |
| Whitmyer headrest adapter | | | R804 | \$55 <input type="checkbox"/> | R804 | \$55 <input type="checkbox"/> |
| Total | | | R840 | \$ | R850 | \$ |

* Required

Room to grow

Large chair at its smallest dimensions

- 1 Backrest height 19"
- 2 Distance between laterals 9½"
- 3 Seat depth 15"
- 4 Seat width 11"
- 5 Armrest height above seat 7"
- 6 Seat height above footboard 11¼"

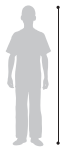


Large chair at its largest dimensions

- 1 Backrest height 24"
- 2 Distance between laterals 14"
- 3 Seat depth 20"
- 4 Seat width 17"
- 5 Armrest height above seat 10½"
- 6 Seat height above footboard 21¼"

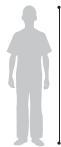


Large standard base
R860 Rifton Activity Chair

| User dimensions (inches) | |
|---|---|
| Height | 50–74 |
|  | Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth. |
| | Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended. |
| Item dimensions (inches) | |
| Frame width | short legs: 26¾ |
| | long legs: 26¾ |
| | short legs w/ casters: 27¾ |
| | long legs w/ casters: 27¾ |
| Seat height above floor | short legs: 16½–20½ |
| | long legs: 20–24 |
| | short legs w/ casters: 16½–20½ |
| | long legs w/ casters: 20–24 |
| Seat angle (tilt-in-space)–degrees | 15° forward, 15° back |
| Backrest angle–degrees | 5° forward, 20° back |
| Footboard knee angle–degrees | 45°– 110° |
| Seat height above footboard | 11¼–21¼ |
| Seat width with hip guides without hip guides | 11–14 |
| | 17 |
| Seat depth | 15–20 |
| Armrest height above seat | 7–10½ |
| Distance between laterals | 9½–14 |
| Backrest height | 19–24 |
| Headrest height above seat | 19½–29½ |
| Max. working load (lb) | 250 |



Large hi/lo base
R870 Rifton Activity Chair

| User dimensions (inches) | |
|---|---|
| Height | 50–74 |
|  | Key user dimension: height The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth. |
| | Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended. |
| Item dimensions (inches) | |
| Frame width | 29¾ |
| Seat height above floor | 16–24 |
| Seat angle (tilt-in-space)–degrees | 15° forward, 25° back |
| Backrest angle–degrees | 5° forward, 20° back |
| Footboard knee angle–degrees | 45°– 110° |
| Seat height above footboard | 11¼–21¼ |
| Seat width with hip guides without hip guides | 11–14 |
| | 17 |
| Seat depth | 15–20 |
| Armrest height above seat | 7–10½ |
| Distance between laterals | 9½–14 |
| Backrest height | 19–24 |
| Headrest height above seat | 19½–29½ |
| Max. working load (lb) | 250 |

2021 order form for large

Use dimension chart to select appropriate size.



| | | Standard R860 large | Hi/lo R870 large | |
|--|--------|-------------------------------------|-------------------------------------|---------------------------------------|
| * Seat and back includes seatbelt (Hi/lo includes footboard and push handles) | CHOOSE | Adjustable | R861 \$660 <input type="checkbox"/> | R871 \$1,260 <input type="checkbox"/> |
| | | Adjustable w/ back spring | R862 \$765 <input type="checkbox"/> | R872 \$1,365 <input type="checkbox"/> |
| * Base | CHOOSE | Without spring | R873 \$405 <input type="checkbox"/> | R883 \$2,730 <input type="checkbox"/> |
| | | With spring | R874 \$485 <input type="checkbox"/> | N/A |
| * Arm supports (pair) | CHOOSE | Armrests | R868 \$235 <input type="checkbox"/> | R868 \$235 <input type="checkbox"/> |
| | | Forearm prompts | R879 \$610 <input type="checkbox"/> | R879 \$610 <input type="checkbox"/> |
| * Adjustable legs | CHOOSE | Short | R808 \$65 <input type="checkbox"/> | N/A |
| | | Long | R894 \$75 <input type="checkbox"/> | N/A |
| | | Short with casters | R895 \$285 <input type="checkbox"/> | N/A |
| | | Long with casters | R896 \$310 <input type="checkbox"/> | N/A |
| * Pads | CHOOSE | Tan | R875 \$245 <input type="checkbox"/> | R875 \$245 <input type="checkbox"/> |
| | | Blue | R876 \$245 <input type="checkbox"/> | R876 \$245 <input type="checkbox"/> |
| | | Red | R877 \$245 <input type="checkbox"/> | R877 \$245 <input type="checkbox"/> |
| | | Pink | R977 \$245 <input type="checkbox"/> | R977 \$245 <input type="checkbox"/> |
| | | Purple | R978 \$245 <input type="checkbox"/> | R978 \$245 <input type="checkbox"/> |
| | | Green | R979 \$245 <input type="checkbox"/> | R979 \$245 <input type="checkbox"/> |
| Headrest | CHOOSE | Contoured | R897 \$230 <input type="checkbox"/> | R897 \$230 <input type="checkbox"/> |
| | | Flat | R898 \$215 <input type="checkbox"/> | R898 \$215 <input type="checkbox"/> |
| | | Adjustable winged | R881 \$285 <input type="checkbox"/> | R881 \$285 <input type="checkbox"/> |
| Laterals (pair) | CHOOSE | Small | R828 \$190 <input type="checkbox"/> | R828 \$190 <input type="checkbox"/> |
| | | Large | R838 \$205 <input type="checkbox"/> | R838 \$205 <input type="checkbox"/> |
| | | Large with strap | R839 \$245 <input type="checkbox"/> | R839 \$245 <input type="checkbox"/> |
| Push handles (recommended for all chairs with wheels) | | R878 \$145 <input type="checkbox"/> | Included | |
| Tray (requires armrests) | | R669 \$275 <input type="checkbox"/> | R669 \$275 <input type="checkbox"/> | |
| Handhold | | R649 \$25 <input type="checkbox"/> | R649 \$25 <input type="checkbox"/> | |
| Chest strap (wide) | | R807 \$90 <input type="checkbox"/> | R807 \$90 <input type="checkbox"/> | |
| Butterfly harness | CHOOSE | Regular | R889 \$140 <input type="checkbox"/> | R889 \$140 <input type="checkbox"/> |
| | | Slim cut | R826 \$140 <input type="checkbox"/> | R826 \$140 <input type="checkbox"/> |
| Thigh belt | | R813 \$90 <input type="checkbox"/> | R813 \$90 <input type="checkbox"/> | |
| Pelvic harness | | R846 \$170 <input type="checkbox"/> | R846 \$170 <input type="checkbox"/> | |
| Hip guides (pair) | | R867 \$200 <input type="checkbox"/> | R867 \$200 <input type="checkbox"/> | |
| Abductor | CHOOSE | Small | R884 \$120 <input type="checkbox"/> | R884 \$120 <input type="checkbox"/> |
| | | Large | R891 \$140 <input type="checkbox"/> | R891 \$140 <input type="checkbox"/> |
| Adductors (pair) | | R892 \$190 <input type="checkbox"/> | R892 \$190 <input type="checkbox"/> | |
| Leg prompt | | R859 \$225 <input type="checkbox"/> | R859 \$225 <input type="checkbox"/> | |
| Footboard (recommended for chairs with long legs) | | R857 \$470 <input type="checkbox"/> | Included | |
| Ankle straps (pair) | | R855 \$90 <input type="checkbox"/> | R855 \$90 <input type="checkbox"/> | |
| Sandals (pair) | | R964 \$105 <input type="checkbox"/> | R964 \$105 <input type="checkbox"/> | |
| Wedges (pair, require sandals) | | R699 \$70 <input type="checkbox"/> | R699 \$70 <input type="checkbox"/> | |
| Lumbar and seat support kit | | R888 \$85 <input type="checkbox"/> | R888 \$85 <input type="checkbox"/> | |
| Backrest filler pad | | R856 \$120 <input type="checkbox"/> | R856 \$120 <input type="checkbox"/> | |
| Whitmyer headrest adapter | | R804 \$55 <input type="checkbox"/> | R804 \$55 <input type="checkbox"/> | |
| Total | | R860 \$ _____ | R870 \$ _____ | |

* Required