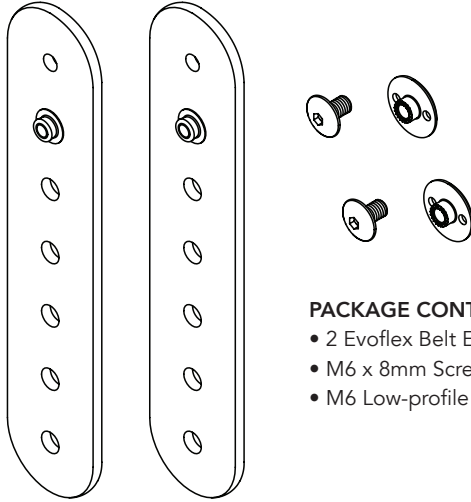
Evoflex Belt Extender

INSTALLATION AND USER'S INSTRUCTIONS

ART NO. EBE100

PACKAGE CONTENTS

- 2 Evoflex Belt Extenders
- M6 x 8mm Screws
- M6 Low-profile flange nuts

These instructions provide important information for the safe use and maintenance of all Bodypoint Evoflex Belt Extenders. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

Optional Bodypoint belt mounting hardware may be used for mounting to wheelchairs and seating systems.

This product is intended for mounting positioning belts and harnesses. It is NOT intended for use as a transportation safety device.

Warranty: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

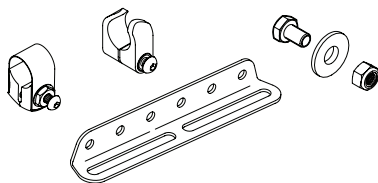
For more information on Bodypoint products and a list of distributors outside the USA go to www.bodypoint.com.

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
MOUNTING OPTIONS (Purchased separately) Belt Extenders can be attached to the wheelchair using Band Clamps (Art No. HW320), Frame Clamps (Art No. HW310), Seat Mounting Brackets (Art No. HW200), or T-Slot Fasteners (Art Nos. HKA6 & HKA10).


INSTALLATION

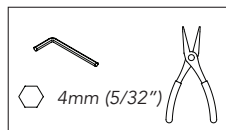
Attach belt extender to Evoflex.
Spring washer not used.

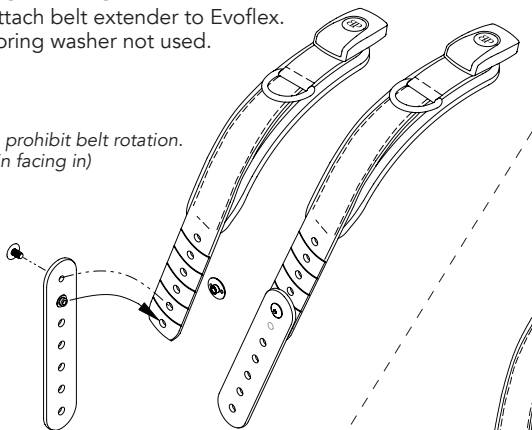
To prohibit belt rotation.
(Pin facing in)

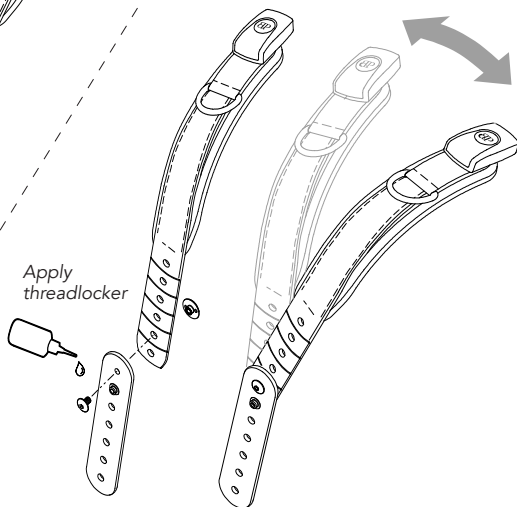To allow belt rotation. (Pin facing out)


Apply
threadlocker

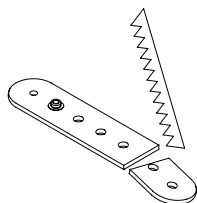
Cut to length as needed using a saw. Ensure cut location is as far away as possible from mounting hole being used.

MOUNTING

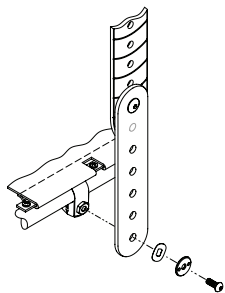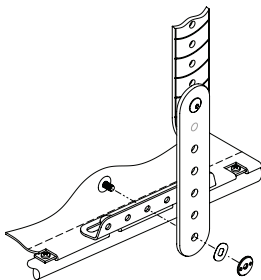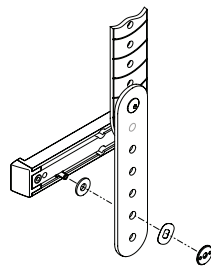
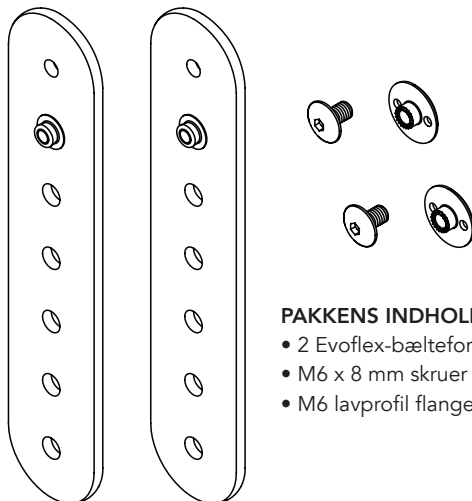
Attach Belt Extender with Evoflex to wheelchair.
Refer to Evoflex product instructions for more detail.
(Spring washer optional)

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