
Rev B



Rotate Evoflex and Belt Extender Around an Obstacle

When to use: to rotate the Evoflex out of the way when an obstacle blocks the Belt Extender from rotating.

- 1. Place 1-2 drops of threadlocker (e.g., Loctite) on the truss head screw.
- 2. Install Belt Extender with locking pin facing away from Evoflex.
- 3. Mount to the chair—the Belt Extender may be blocked, but the Evoflex can rotate out of the way.

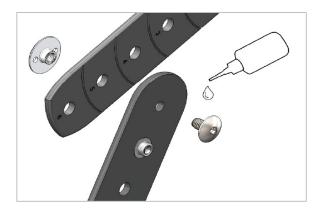



Bend Belt Extender with Heat

When to use: to mount Belt Extender under solid seat bases or around back supports.

- 1. Measure and mark bend line. Ideal bend line is between two mounting holes. Be sure to mark the correct side, considering whether the locking pin will be used or not.
- 2. Apply heat directly to bend line using a heat gun at 120-150°C (250-300°F), or immerse in boiling water for 30 seconds.
- 3. Bend over a form, such as a wood block, to control bend location and angle.
- 4. Belt Extender can be reheated and rebent as needed, as long as the maximum temperature of 150°C (300°F) is not exceeded.





Cut Belt Extender with Saw

When to use: to shorten Belt Extender.

- 1. Measure and mark cut line. Ideal cut line is between two mounting holes.
- 2. Secure Belt Extender in a vise with flat side up.
- 3. Cut to length using hacksaw. Belt Extender can also be cut with bandsaw.
- 4. File or sand rough edges to smooth.

