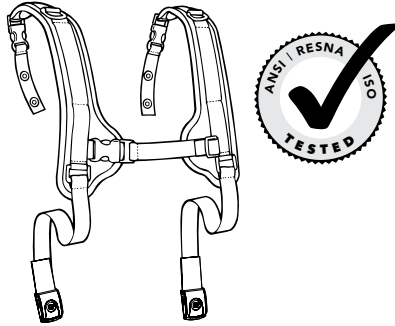
H-Style Shoulder Harness

ART NOS. SH210, SH220

These instructions provide important information for the safe use and maintenance of all Bodypoint H-Style Shoulder Harnesses. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

⚠ WARNING! Product should be installed and fitted by a qualified rehab technician.

⚠ WARNING! Because of the risk of choking, it is dangerous to use this product without stabilizing the pelvis – always use with a properly fitted pelvic support belt.

For installation, this product should be mounted with:

- Flat-Mount end-fittings
- Cinch-Mount end-fittings
- Cam buckle (suitable for users up to 75kg)

Optional Bodypoint belt mounting hardware may be used for mounting to wheelchairs and seating systems.

USE:

⚠ WARNING! This product should only be used for positioning a person in a wheelchair. It is NOT intended for use as a transportation safety device, as a personal restraint device, or in any other application where its failure could result in injury. Misuse of this product is unauthorized and unsafe.

⚠ WARNING! Accidental release of this shoulder harness can allow the user to fall forward. If the user's movements or cognitive abilities could lead to accidental release, a caregiver must be present at all times during its use. Ensure that all caregivers know how to unfasten the product. Failure to do so may delay release in an emergency.

⚠ WARNING! As with any new seating support, this product may change the way a person sits. Users must continue to practice regular pressure relief activities and skin integrity checks, not only where this product contacts the user, but also in primary pressure-bearing areas such as the sacrum, legs, and buttocks. If increased skin redness or irritation occurs, discontinue use and consult your physician or seating specialist. Failure to do so may result in serious injury, such as pressure ulcers.

MAINTENANCE: Check periodically for signs of wear in the stitching, webbing, and pads. If significant wear is found, contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons—health and safety depend on it!

🧺🔥🧺🧺 CLEANING: Machine wash, hot, 60°C (140°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

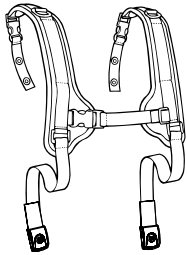
For more information on Bodypoint products, and a list of distributors outside the USA, go to www.bodypoint.com

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CE


PACKAGE CONTENTS

- Shoulder Harness
- Cinch-Mount™ or Flat-Mount™ End Fittings
- Cam Buckles (Optional –Art. No. FS032-2)


INSTALLATION

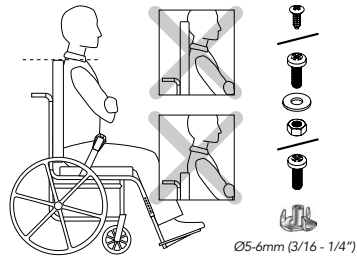Seat user in wheelchair and establish correct posture with **Pelvic Support Belt**: Place harness on user with upper straps between neck and shoulder joint.Center the sternum strap on the sternum.



INSTALLATION REQUIREMENTS

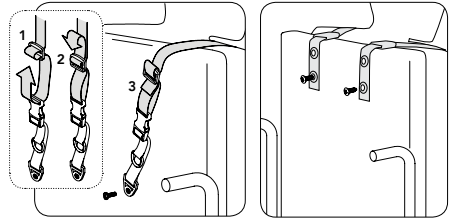1. Wheelchair with a solid backrest or strap guides set approximately at shoulder height (adjust up/down for pressure distribution).
2. Mounting screws with a minimum pullout strength of 200lbs/90kg. Not included in package.



TOP STRAPS

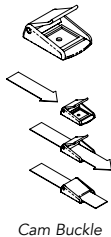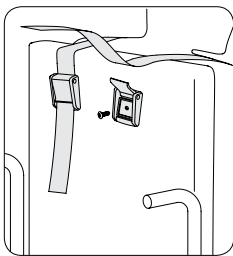1. Adjust each shoulder strap for a comfortable fit, according to the user's shoulder width and sitting position. Mark for mounting screws to hold this position.
2. Attach to backrest using suitable fasteners.

DIRECT MOUNTING
Rear-Pull versions only. Attach Cam Buckle to backrest using suitable fasteners. Thread webbing through buckle. Suitable for users up to 75kg.

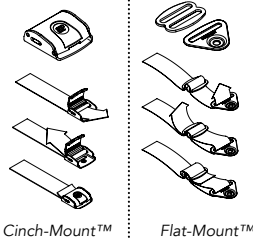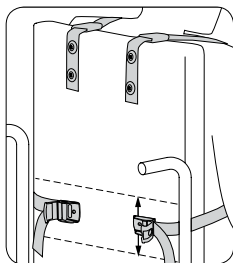Cam Buckle

Cinch-Mount™

Flat-Mount™

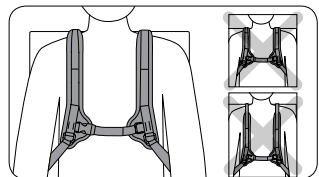
LOWER STRAPS

Wrap lower straps around backrest without obstructing other devices on wheelchair. Mount end fittings above top of the pelvis, between middle and lower back height. Attach to backrest using suitable fasteners.

FITTING

Position the user in wheelchair and fasten the pelvic support belt securely, then buckle the shoulder harness sternum strap, and adjust sternum width Adjust the lower straps to prevent the sternum strap from riding-up. Then tighten the top straps to position the pads and to properly support the user.


SAFETY CHECK

When properly positioned in wheelchair, have user lean forward and side to side to check fit. Check for:

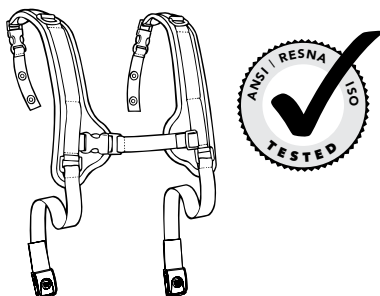1. Normal operation of buckles and adjustment straps.
2. Comfort If rubbing neck, relocate upper straps.
3. Position If too high or too low adjust lower straps.
4. Interference with other devices Relocate straps as necessary to clear seat tilting mechanism, armrests, accessory pads, or feeding tubes.

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